

CLR[©]

CLR-online.com

CLINICAL LABORATORY REFERENCE

2023-2024

FEATURING:

**Over 4,900 tests
Equipment
Products
Services**



REFERENCE GUIDES:

**Critical limits
Cutoff concentrations
Reference intervals
Drug levels**

A SUPPLEMENT OF:

ML
MEDICAL LABORATORY OBSERVER[®]



COLLEGE of AMERICAN PATHOLOGISTS

Laboratory Quality Solutions

We help you improve laboratory quality every day.

Built on a foundation of pathologist expertise, the College of American Pathologists (CAP) partners with laboratories worldwide to elevate the quality of laboratory medicine with best-in-class solutions designed to drive operational excellence, achieve diagnostic confidence, and ensure the best patient care.

LABORATORY ACCREDITATION PROGRAM

Improve quality and reduce risk through customized, discipline-specific checklists, collegial peer inspections, and educational resources.



PROFICIENCY TESTING (PT)

Ensure accurate and reliable testing results with more than 700 programs across 16 disciplines developed by expert scientific committees.

PROTOCOLS AND GUIDELINES

Improve cancer reporting and clinical decision making, providing more effective testing with consistent, high-quality results and expert interpretations.

QUALITY IMPROVEMENT

Improve processes and strengthen staff expertise while reducing risk and improving patient outcomes.


Learn more about how the CAP can help you
achieve your laboratory quality goals.



2023-2024

CLR[®]

CLINICAL LABORATORY REFERENCE

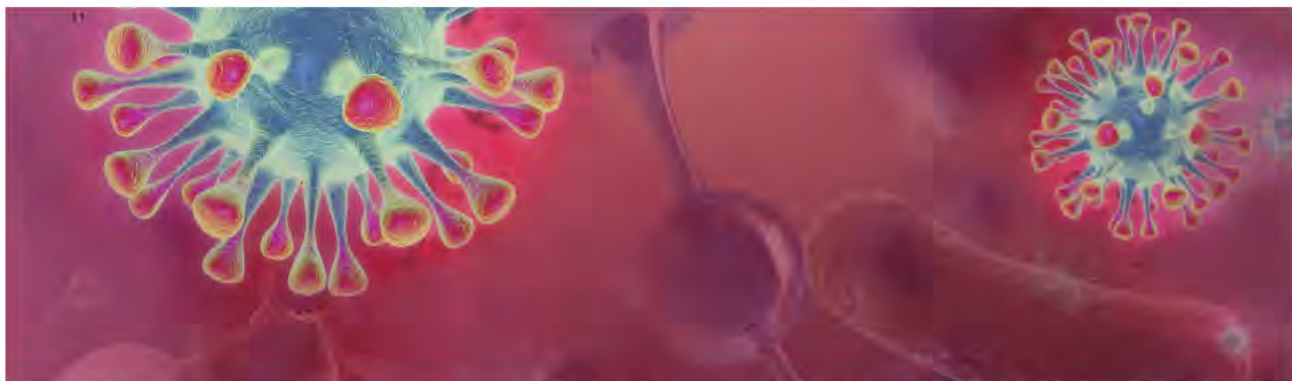

140965018 © Sjarhei Yurchanka | 170999814 © Romolo Tavani | 18468385 © Heromen30 | Dreamstime.com

FEATURES

CLINICAL LABORATORY REFERENCE GUIDES

- 06 Table of Critical Limits
- 08 Cutoff Concentrations for Drug Tests
- 09 Table of Reference Intervals
- 10 Critical Values for Therapeutic Drug Levels

PRODUCT LISTINGS AND TESTS

- 12 Clinical Test and Product Information
- 24 Index of Tests, Equipment, and Services

DEPARTMENTS

- 02 From the editor
- 04 Special Feature: 2023 diagnostic updates
By MLO Staff

HOW TO USE CLR

REFERENCE GUIDES

CLR is an annual supplement provided by MLO reflecting peer-reviewed clinical laboratory reference guides, as well as market resources available to clinical laboratorians.

PRODUCT INFORMATION

The product information section includes company descriptions, their essential laboratory products, and contact information for pricing and ordering.

INDEX OF TESTS, EQUIPMENT, AND SERVICES

The alphabetical index conveniently categorizes and cross-references laboratory products by test names, equipment types, and services provided.

READER FEEDBACK

Visit CLR-online.com to access the digital version. Subscribe to receive future CLR supplements here: <https://www.mlo-online.com/subscribe/>

MLO's annual reference guide



Medical Laboratory Observer has published its annual reference guide, *Clinical Laboratory Reference (CLR)*, since 1972. *CLR* provides a guide to tests, equipment, and services for the clinical lab market, in addition to tables of critical limits, reference intervals, and critical values. It is the industry's only complete resource guide.

The *CLR* product guide includes company descriptions, their essential laboratory products, and contact information for ordering and pricing. This is a "Yellow Pages" service for our readers.

Critical limits and reference intervals help laboratory professionals interpret test results. Critical limits establish the upper and lower values that indicate when a patient's medical condition may be precarious, requiring the immediate attention of the provider who ordered the test. Reference intervals are the most common decision support tool for interpreting pathology reports, as patients' test results are compared to these ranges to indicate normal results or those that may require clinical follow-up. These decision tools assist in one of the top goals in healthcare: timely communication.

Also in this issue, MLO editors pulled together summaries of some of the biggest diagnostic updates from the U.S. Food and Drug Administration and National Institutes of Health so far in 2023. One of these updates is deeper insight into long COVID — the post-infection set of conditions that can affect nearly every tissue and organ in the body. Researchers, funded by the National Institutes of Health, have identified more than 200 symptoms associated with long COVID. Clinical symptoms vary but include fatigue, brain fog, dizziness, loss of taste and smell, and they can last for months or years after a person has had COVID-19. The research team also found that long COVID was more common and severe in study participants infected before the 2021 Omicron variant. To date, more than 100 million Americans have been infected with SARS-CoV-2, the virus that causes COVID-19. It is estimated that about 10% of adults infected with the virus continue to experience and suffer from the many symptoms termed together as long COVID.

As is the case every year, the editors at MLO would not have been able to produce this year's *CLR* issue without the help of experts such as the following:

- Gerald J. Kost, MD, PhD, MS, FAACC, Emeritus Professor, University of California, Davis for the Table of Critical Limits.
- Sean Campbell, PhD, DABCC, FAACC, Director of Chemistry, Montefiore Medical Center, Bronx, NY for the Table of Reference Intervals.
- Rajasri Chandra, MS, MBA for the Critical Values for Therapeutic Drug Values.

We hope our readers find this resource beneficial. For the online version of *CLR*, please visit www.clr-online.com.

For comments or feedback on *CLR*, please feel free to reach out to me at cwichmann@mlo-online.com.

VP, Group Publisher
Matthew Raynor
mraynor@endeavorb2b.com

Editor in Chief
Christina Wichmann
cwichmann@mlo-online.com

Managing Editor
Erin Brady
erbrady@endeavorb2b.com

Production Manager
Edward Bartlett
ebartlett@endeavorb2b.com

Art Director
Patti Connors
pconnors@endeavorb2b.com

Audience Development/List Rentals
Laura Moulton
lmoulton@endeavorb2b.com

Advertising Services Manager
Karen Runion
krunion@endeavorb2b.com

ADVERTISING

Director of Sales
East Coast/Midwest Sales, Classifieds

Carol Vovcsko
(941) 321-2873
covovcsko@mlo-online.com

South/West Coast/Illinois Sales

Lora Harrell
(941) 328-3707
lharrell@mlo-online.com

MLO EDITORIAL ADVISORY BOARD

John Brunstein, PhD, Biochemistry
(Molecular Virology)

President & CEO
PathoID, Inc., British Columbia, Canada

Lisa-Jean Clifford
COO & Chief Strategy Officer
Gestalt, Spokane, WA

Barbara Strain, MA, SM(ASCP), CVAHP
Principal, Barbara Strain Consulting LLC
Formerly Director, Value Management
University of Virginia Health System, Charlottesville, VA

Jeffrey D. Klausner, MD, MPH
Professor of Preventive Medicine in the Division of Disease Prevention, Policy and Global Health, Department of Preventive Medicine at University of Southern California
Keck School of Medicine

Donna Beasley, DLM(ASCP)

Director
Huron Healthcare, Chicago, IL
Anthony Kurec, MS, H(ASCP)DLM
Clinical Associate Professor, Emeritus

SUNY Upstate Medical University, Syracuse, NY
Suzanne Butch, MLS(ASCP)CM, SBB^{CM}, DLM^{CM}
Freelance Consultant, Avon, OH

Paul R. Eden, Jr., MT(ASCP), PhD
Lt. Col., USAF (ret.)
(formerly) Chief, Laboratory Services
88th Diagnostics/Therapeutics Squadron
Wright-Patterson AFB, OH

Daniel J. Scungio, MT(ASCP), SLS, CQA (ASQ)
Consultant at Dan the Lab Safety Man and Safety Officer
at Sentara Healthcare, Norfolk, VA

CORPORATE TEAM



CEO Chris Ferrell

CFO Mark Zedell

President June Griffin

COO Patrick Rains

CRO Reggie Lawrence

Chief Digital Officer Jacqui Niemiec

Chief Administrative and Legal Officer Tracy Kane

Amy Mularski, EVP City Services & Healthcare

22477 Stickney Point Rd., Suite 221B Sarasota, FL 34231

Phone: (941) 388-7050 Fax: (941) 388-7490

www.mlo-online.com

Medical Laboratory Observer USPS Permit 80930, ISSN 0590-7247 print, ISSN 2771-6759 online is published 12 times annually (Jan, Feb, Mar, Apr, May, Jun, Jul, Aug, Sep, Oct, Nov, Dec), with an additional issue in August, by Endeavor Business Media, LLC, 1233 Janesville Ave., Fort Atkinson, WI 53538. Periodical postage paid at Fort Atkinson, WI, and additional mailing offices. POSTMASTER: Send address changes to Medical Laboratory Observer, PO Box 3257, Northbrook, IL 60065-3257. SUBSCRIPTIONS: Publisher reserves the right to reject non-qualified subscriptions. Subscription prices: U.S. \$180.00 per year; Canada/Mexico \$193.75 per year; All other countries \$278.25 per year. All subscriptions are payable in U.S. funds. Send subscription inquiries to Medical Laboratory Observer, PO Box 3257, Northbrook, IL 60065-3257. Customer service can be reached toll-free at 877-382-9187 or at MLO@meda.com for magazine subscription assistance or questions.

Printed in the USA. Copyright 2023 Endeavor Business Media, LLC. All rights reserved. No part of this publication may be reproduced or transmitted in any form or by any means, electronic or mechanical, including photocopies, recordings, or any information storage or retrieval system without permission from the publisher. Endeavor Business Media, LLC does not assume and hereby disclaims any liability to any person or company for any loss or damage caused by errors or omissions in the material herein, regardless of whether such errors result from negligence, accident, or any other cause whatsoever. The views and opinions in the articles herein are not to be taken as official expressions of the publishers, unless so stated. The publishers do not warrant either expressly or by implication, the factual accuracy of the articles herein, nor do they so warrant any views or opinions by the authors of said articles.



Autoimmune Diagnostics

For more than 35 years, KRONUS has provided specialized immunoassay test kits to medical professionals at the world's most respected laboratory facilities. Our current product offering encompasses test kits for measurement of the following:

NEUROIMMUNOLOGY

Aquaporin-4 Autoantibody (AQP4Ab)
Acetylcholine Receptor Antibody (AChRAB):

- Binding Antibody
- Blocking Antibody

Voltage-Gated Calcium Channel Antibody: P/Q-Type

ISLET CELL AUTOIMMUNITY

Glutamic Acid Decarboxylase Antibody (GADAb)
Zinc Transporter 8 Autoantibody (ZnT8Ab)
IA-2 Autoantibody (IA-2Ab)
Insulin Autoantibody (IAA)

THYROID

TSH Receptor Antibody (TRAb)

ADRENAL AUTOIMMUNITY

21-Hydroxylase Antibody (21-OHAb)

To obtain additional information on **KRONUS'** unique line of laboratory test kits, please call us toll-free at **800 4 KRONUS** or email us at **kronus@kronus.com**.

ALSO AVAILABLE FOR RESEARCH APPLICATIONS

Acetylcholine Receptor Antibody: Ganglionic (gAChRAB)[†]
Voltage-Gated Calcium Channel Antibody: N-Type (N-VGCCAb)[†]
GAD/IA-2/ZnT8 Antibody Screen[†]
Titin Antibody (TitinAb)[†]
Voltage-Gated Potassium Channel Antibody (VGKCAb)[†]

[†] For Research Use Only.
Not For Use in Diagnostic Procedures.

2023 diagnostic updates

By MLO staff

Medical Laboratory Observer connects laboratory professionals with industry trends every year. There have been important diagnostic updates for laboratories from the U.S. Food and Drug Administration and the National Institutes of Health in 2023 — MLO staff put this article together to summarize those updates for its readers.

With this new research, medical experts are closer to identifying better cancer treatments, determining blood donor eligibility, having a deeper understanding of long COVID, and diagnosing severe COVID-19 in pediatric patients.

FDA launches pilot program to help reduce risks associated with using laboratory developed tests to identify cancer biomarkers

The U.S. Food and Drug Administration announced a new voluntary pilot program for certain oncology drug products used with certain corresponding in vitro diagnostic tests to help clinicians select appropriate cancer treatments for patients.

Through the pilot program, the FDA will request, from drug manufacturers, performance information for the tests used to enroll patients into the clinical trials that support drug approval. Based on an assessment of that information, the FDA will post to the FDA website the minimum performance characteristics recommended for similar tests that may be used to select patients for treatment with the approved drug. Laboratories may use this information to guide their development of LDTs to identify specific biomarkers used for selecting cancer treatment. This transparency aims to help facilitate better and more consistent performance of these tests, resulting in better drug selection and improved care for patients with cancer.

As discussed in the guidance, the initial phase of the pilot program is anticipated to last up to one year, during which the FDA will evaluate no more than nine drug sponsors for possible acceptance into the pilot. The minimum recommended performance characteristics for in vitro diagnostic tests used with each approved drug product under the pilot, based on the clinical trial assays, will be made publicly available on the FDA's website following drug approval.¹

FDA finalizes move to recommend individual risk assessment to determine eligibility for blood donations

The U.S. Food and Drug Administration finalized recommendations for assessing blood donor eligibility using a set of individual risk-based questions to reduce the risk of transfusion-transmitted HIV. These questions will be the same for every donor, regardless of sexual orientation, sex, or gender. Blood establishments may now implement these recommendations by revising their donor history questionnaires and procedures.

These final recommendations are consistent with the policy initially proposed in January 2023.

This policy eliminates time-based deferrals and screening questions specific to men who have sex with men (MSM) and women who have sex with MSM. Under the final guidance issued, all prospective blood donors will answer a series of individual, risk-based questions to determine eligibility. All prospective donors who report having a new sexual partner, or more than one sexual partner in the past three months, and



94996542 © Mast3r | Dreamstime.com

anal sex in the past three months, would be deferred to reduce the likelihood of donations by individuals with new or recent HIV infection who may be in the window period for detection of HIV by nucleic acid testing.

Additionally, under these final recommendations, those taking medications to treat or prevent HIV infection (e.g., antiretroviral therapy (ART), pre-exposure prophylaxis (PrEP) and post-exposure prophylaxis (PEP)), will also be deferred. Though these antiretroviral drugs are safe, effective, and an important public health tool, the available data demonstrate that their use may delay detection of HIV by currently licensed screening tests for blood donations, which may potentially give false negative results. Although HIV is not transmitted sexually by individuals with undetectable viral levels, this does not apply to transfusion transmission of HIV because a blood transfusion is administered intravenously, and a transfusion involves a large volume of blood compared to exposure with sexual contact. As stated in the guidance, individuals should not stop taking their prescribed medications, including PrEP, or PEP, in order to donate blood.²

Large study provides scientists with deeper insight into long COVID symptoms

Initial findings from a study of nearly 10,000 Americans, many of whom had COVID-19, have uncovered new details about long COVID, the post-infection set of conditions that can affect nearly every tissue and organ in the body. Clinical symptoms can vary and include fatigue, brain fog, and dizziness, and last for months or years after a person has COVID-19. The research team, funded by the National Institutes of Health, also found that long COVID was more common and severe in study participants infected before the 2021 Omicron variant.

The study, published in *JAMA*, is coordinated through the NIH's Researching COVID to Enhance Recovery (RECOVER) initiative, a nationwide effort dedicated to understanding why some people develop long-term symptoms following COVID-19, and most importantly, how to detect, treat, and prevent long COVID. The researchers hope this study is the next step toward

potential treatments for long COVID, which affects the health and wellbeing of millions of Americans.

Researchers examined data from 9,764 adults, including 8,646 who had COVID-19 and 1,118 who did not have COVID-19. They assessed more than 30 symptoms across multiple body areas and organs and applied statistical analyses that identified 12 symptoms that most set apart those with and without long COVID: post-exertional malaise, fatigue, brain fog, dizziness, gastrointestinal symptoms, heart palpitations, issues with sexual desire or capacity, loss of smell or taste, thirst, chronic cough, chest pain, and abnormal movements.

They then established a scoring system based on patient-reported symptoms. By assigning points to each of the 12 symptoms, the team gave each patient a score based on symptom combinations. With these scores in hand, researchers identified a meaningful threshold for identifying participants with long COVID. They also found that certain symptoms occurred together and defined four subgroups or “clusters” with a range of impacts on health.

Based on a subset of 2,231 patients in this analysis who had a first COVID-19 infection on or after Dec. 1, 2021, when the Omicron variant was circulating, about 10% experienced long-term symptoms or long COVID after six months. The results are based on a survey of a highly diverse set of patients and are not final. Survey results will next be compared for accuracy against an array of lab tests and imaging.

The researchers explain studying the underlying biological mechanisms of long COVID is central to advancing informed interventions and identifying effective treatment strategies.

In addition to establishing the scoring system, the researchers found that participants who were unvaccinated or who had COVID-19 before the Omicron strain emerged in 2021 were more likely to have long COVID and more severe cases of long COVID. Further, reinfections were also linked to higher long COVID frequency and severity, compared to people who only had COVID-19 once.³

NIH funds eight studies to advance rapid diagnosis of COVID-19–related inflammatory syndrome in children

The National Institutes of Health has awarded eight research grants to refine new technologies for early diagnosis of severe illnesses resulting from SARS-CoV-2 infection in children. The new awards follow grants issued in 2020 to foster methods for diagnosing children at high risk for Multisystem Inflammatory Syndrome in Children (MIS-C), a rare, severe and sometimes fatal after-effect of SARS-CoV-2 infection or exposure in children.

The awards are from NIH’s Predicting Viral-Associated Inflammatory Disease Severity in Children with Laboratory Diagnostics and Artificial Intelligence (PreVAIL kIDs) initiative. They are part of the Rapid Acceleration of Diagnostics Radical (RADx-rad) program to support new, non-traditional approaches and reimaged uses of existing tools to address gaps in COVID-19 testing and surveillance.

Although some children develop mild or no symptoms from COVID-19, others will develop more severe effects, including MIS-C, which results in inflammation of one or more organs, including the heart, lungs, kidneys, brain, skin, eyes and gastrointestinal tract.

The 2020 awards supported studies involving more than 7,400 research participants in four countries and yielded prototype methods and techniques for potential use in clinics, emergency departments and for hospital inpatients. These PreVAIL kIDs studies were supported through NIH’s RADx-rad

initiative and were part of an NIH collaborative research effort called CARING for Children with COVID. Results from these studies include a laboratory technique for detecting specific immune cells associated with MIS-C; databases that help diagnose children at risk for MIS-C and severe COVID-19, based on certain blood proteins and genetic biomarkers; and a database that can distinguish between MIS-C, Kawasaki disease (which has similar symptoms) and fever-causing viral and bacterial infections.

The new awards will allow researchers to continue their efforts to develop ways to rapidly diagnose MIS-C and identify those at risk for serious and long-term effects of SARS-CoV-2. Earlier identification of those most at risk will allow for earlier interventions to prevent severe health effects.

Awardees

- Jane C. Burns, University of California, San Diego
Diagnosing and predicting risk in children with SARS-CoV-2-related illness
- Cedric Manlhot, Johns Hopkins University, Baltimore
Data science approach to MIS-C identification and management associated with SARS
- Ananth V. Annapragada, Baylor College of Medicine, Houston-AICORE-kids: Artificial intelligence COVID-19 risk assessment for kids
- Audrey R. Odom John, Children’s Hospital of Philadelphia
Diagnosis of MIS-C in febrile children
- Usha Sethuraman, Central Michigan University, Mount Pleasant
Severity predictors integrating salivary transcriptomics and proteomics with multineural network intelligence in SARS-CoV2 infection in children
- Juan C. Salazar, Connecticut Children’s Medical Center, Hartford
Identifying biomarker signatures of prognostic value for MIS-C
- Charles Yen Chiu, University of California, San Francisco
Discovery and clinical validation of host biomarkers of disease severity and MIS-C with COVID-19
- Lawrence Kleinman, Rutgers Robert Wood Johnson Medical School, New Brunswick, New Jersey
COVID-19 network of networks expanding clinical and translational approaches to predict severe illness in children⁴

REFERENCES

1. FDA launches pilot program to help reduce risks associated with using laboratory developed tests to identify cancer biomarkers. U.S. Food and Drug Administration. Published June 20, 2023. Accessed June 23, 2023. <https://www.fda.gov/news-events/press-announcements/fda-launches-pilot-program-help-reduce-risks-associated-using-laboratory-developed-tests-identify>.
2. FDA finalizes move to recommend individual risk assessment to determine eligibility for blood donations. U.S. Food and Drug Administration. Published May 11, 2023. Accessed June 23, 2023. <https://www.fda.gov/news-events/press-announcements/fda-finalizes-move-recommend-individual-risk-assessment-determine-eligibility-blood-donations>.
3. Large study provides scientists with deeper insight into long COVID symptoms. National Institutes of Health (NIH). Published May 25, 2023. Accessed June 23, 2023. <https://www.nih.gov/news-events/news-releases/large-study-provides-scientists-deeper-insight-into-long-covid-symptoms>.
4. NIH funds eight studies to advance rapid diagnosis of COVID-19-related inflammatory syndrome in children. National Institutes of Health (NIH). Published January 9, 2023. Accessed June 23, 2023. <https://www.nih.gov/news-events/news-releases/nih-funds-eight-studies-advance-rapid-diagnosis-covid-19-related-inflammatory-syndrome-children>.

Adult

CLINICAL CHEMISTRY					
		LOW LIMIT		HIGH LIMIT	
Test	Units	Mean (SD)	Range	Mean (SD)	Range
Glucose	mmol/L mg/dL	2.6 (0.4) 46 (7)	1.7-3.9 30-70	26.9 (8.0) 484 (144)	6.1-55.5 110-1000
Potassium	mmol/L	2.8 (0.3)	2.5-3.6	6.2 (0.4) 8.0 (hemolyzed)	5.0-8.0
Calcium	mmol/L mg/dL	1.65 (0.17) 6.6 (0.7)	1.25-2.15 5.0-8.6	3.22 (0.22) 12.9 (0.9)	2.62-3.49 10.5-14.0
Sodium	mmol/L	120 (5)	110-137	158 (6)	145-170
CO ₂ content	mmol/L	11 (2)	5-20	40 (3)	35-50
Magnesium	mmol/L mg/dL	0.41 (0.16) 1.0 (0.4)	0.21-0.74 0.5-1.8	2.02 (0.82) 4.9 (2.0)	1.03-5.02 2.5-12.2
Phosphorus	mmol/L mg/dL	0.39 (0.10) 1.2 (0.3)	0.26-0.65 0.8-2.0	2.87 (0.48) 8.9 (1.5)	2.26-3.23 7.0-10.0
Bilirubin	μmol/L mg/dL	— —	— —	257 (86) 15 (5)	86-513 5-30
Chloride	mmol/L	75 (8)	60-90	126 (12)	115-156
Osmolality	mmol/kg	250 (13)	230-280	326 (18)	295-375
Urea nitrogen	mmol/L mg/dL	— —	— —	37.1 (21.1) 104 (59)	14.3-107.1 40-300
Uric acid	μmol/L mg/dL	— —	— —	773 (119) 13 (2)	595-892 10-15
CSF glucose	mmol/L mg/dL	2.1 (0.6) 37 (10)	1.1-2.8 20-50	24.3 (11.4) 438 (206)	13.9-38.9 250-700
Creatinine	μmol/L mg/dL	— —	— —	654 (380) 7.4 (4.3)	177-1326 2.0-15.0
Ionized calcium ⁴	mmol/L mg/dL	0.82 (0.14) 3.29 (0.56)	0.50-1.07 2.00-4.29	1.55 (0.19) 6.21 (0.76)	1.30-2.00 5.21-8.02
Lactate	mmol/L mg/dL	— —	— —	3.4 (1.3) 30.6 (11.7)	2.3-5.0 20.7-45.0
HEMATOLOGY					
Hematocrit	L/L	0.18 (0.05)	0.12-0.30	0.61 (0.06)	0.54-0.80
Hemoglobin	g/L	66 (17)	40-120	199 (27)	170-300
Platelets	×10 ⁹ /L	37 (18)	10-100	910 (147)	555-1000
WBC count	×10 ⁹ /L	2.0 (0.7)	1.0-4.0	37.0 (20.7)	10.0-100.0
PT	s	—	—	27 (9)	14-40
PTT	s	—	—	68 (33)	32-150
Fibrinogen	g/L	0.88 (0.17)	0.50-1.00	7.75 (2.63)	5.00-10.00
BLOOD GASES AND PH					
pCO ₂	mm Hg	19 (3)	9-25	67 (6)	50-80
pH		7.21 (0.06)	7.00-7.35	7.59 (0.03)	7.50-7.65
pO ₂	mm Hg kPa	43 (6) 5.7 (0.8)	30-55 4.0-7.3	— —	— —

Adult table modified with permission by JAMA, Vol. 263, pp. 704-707, 1990. This table was updated by Dr. Gerald Kost in 2023. CSF, cerebrospinal fluid; WBC, white blood cell; PT, prothrombin time; PTT, partial thromboplastin time. Qualitative critical results for adults¹ include the following: For blood bank and immunology—incompatible crossmatch, tests positive for syphilis (RPR or VDRL). For microbiology and parasitology—positive results from Gram stain or in culture from blood, cerebrospinal fluid, or body cavity fluid; positive India ink preparation; positive rapid antigen detection by agglutination tests for *Cryptococcus*, group B streptococci, *Haemophilus influenzae b*, or *Neisseria meningitidis*, positive results from acid-fast bacillus stain or culture; *Salmonella*, *Shigella*, or *Campylobacter* on stool culture; presence of malarial parasites. For clinical microscopy and urinalysis—elevated white blood cell count in CSF; presence of malignant cells, blasts, or microorganisms in CSF or body fluids; combination of strongly positive test results for glucose and for ketones in urine; presence of pathologic crystals (urate, cysteine, leucine, or tyrosine) on urinalysis. For hematology—listed frequently are the presence of blasts on blood smear; new diagnosis or findings of leukemia; presence of sickle cells (or aplastic crisis). Listed occasionally are plasma cells, band cells, atypical lymphocytes, and abnormal reticulocyte count.

Critical limits define the boundaries of life-threatening diagnostic test results. Critical results falling outside high and low critical limits must be reported to clinicians without delay, so the patient can be treated promptly if necessary. Critical value reporting was first implemented by George Lundberg, MD, and published in MLO in 1972. These tables are based on three national surveys by Gerald Kost, MD, PhD, MS, of UC Davis Health. Adapted with permission from his articles,¹⁻³ they summarize critical limits used by 92 responding U.S. medical centers, including 20 trauma centers, and by 39 children's hospitals.

Mean low and mean high figures may be considered critical limits for each test listed. The frequency with which critical limits were listed can be found in the original articles. Dr. Kost conducted an independent national survey to determine ionized calcium critical limits.³ His overview of critical limits and patient outcomes appeared in MLO⁴ followed by calls for national harmonization and standards of care for critical value practices.^{5,6}

The Joint Commission identifies critical values and the need to report critical results timely in National Patient Safety Goals.⁷ Elements of performance comprise: (a) define critical values and develop written procedures for managing critical results, by whom and to whom they are reported, and acceptable lengths of time between resulting and reporting; (b) implement procedures for managing critical results; and (3) evaluate the timeliness of reporting. Laboratories should carefully monitor failed clinician notifications and strive for none.

Surveys and practice reviews are advancing harmonization⁸ of critical values and notification practices in Australia,⁹ Canada,¹⁰ China,¹¹⁻¹⁶ Croatia,¹⁷ Iran,¹⁸⁻²⁰ Italy,^{21,22} Kuwait,²³ Turkey,²⁴ Spain,²⁵ South Africa,²⁶ and the U.S.²⁷⁻²⁸ Understanding quantitative critical limits, qualitative critical values, alternate nomenclature (e.g., "critical-risk results, significant-risk results, and alert thresholds"²⁹), and national norms will be necessary to enable a global standard. Special considerations for newborn and pediatric critical values are necessary because of rapid adaption to the extrauterine demands for physiological changes.

Studies address critical values for cytology,^{29,30} cytogenetics and molecular genetics,³¹ point-of-care glucose,³² virology,³³ and anatomic and surgical pathology.³⁴ One US study addressed false positive critical value results.³⁵ If institutions list COVID-19 tests, then both false positive and false negatives should be of concern, the former for triggering unnecessary isolation and the latter, for spreading disease by those unaware of infection.³⁶ Repeat testing will improve the performance of COVID-19 testing³⁷ but may not be indicated for other critical results.^{19,24,38,39}

Lifesaving diagnostic speed and accurately informed decision-making lead to appropriate therapy in times of human crises.⁴⁰ Clinical laboratories and point-of-care specialists can assign critical values and develop notification practices collaboratively with emergency physicians, hospitalists, and other clinical colleagues to achieve optimal outcomes for patients.

REFERENCES

- Kost GJ. Critical limits for urgent clinician notification at U.S. medical centers. JAMA. 1990;263:704-707.
- Kost GJ. Critical limits for emergency clinician notification at U.S. children's hospitals. Pediatrics. 1991;88:597-603.
- Kost GJ. The significance of ionized calcium in cardiac and critical care. Availability and critical limits at U.S. medical centers and children's hospitals. Arch Pathol Lab Med. 1993;117:890-896.
- Kost GJ. Using critical limits to improve patient outcomes. MLO. 1993;25(3):22-27.
- Kost GJ. Co-creating critical limits for enhanced acute care: proven need and web knowledge base. Part 1: A call to action! MLO. 2015;47(12):34, 36-37.



6. Kost GJ. Co-creating critical limits for enhanced acute care: proven need and web knowledge base. Part 2: Standard of care, what it means and how it is applied. *MLD*. 2016;48(1):28-29.
7. The Joint Commission. NPSG.02.03.01 for the Laboratory Program, January 2023. Report critical results of tests and diagnostic procedures on a timely basis. Accessed April 18, 2023. <https://www.jointcommission.org/>.
8. Kost GJ, Hale KN. Global trends in critical values practices and their harmonization. *Clin Chem Lab Med*. 2011;49(2):167-176.
9. Campbell C, Horvath A. Towards harmonization of critical laboratory result management—review of the literature and survey of Australian practices. *Clin Biochem Rev*. 2012;33(4):149-160.
10. Gong Y, Adeli K. A national survey on pediatric critical values used in clinical laboratories across Canada. *Clin Biochem*. 2009;42(16-17):1610-1615.
11. Zeng R, Wang W, Wang Z. National survey on critical values notification of 599 institutions in China. *Clin Chem Lab Med*. 2013;51(11):2099-2107.
12. Fei Y, Zhao H, Wang W, He F, Zhong K, Yuan S, et al. National survey on current situation of critical value reporting in 973 laboratories in China. *Biochem Med (Zagreb)*. 2017;27(3):030707.
13. Ye YY, Zhao HJ, Fei Y, Wang W, He FL, Zhong K, et al. Critical values in hematology of 862 institutions in China. *Int J Lab Hematol*. 2017;39(5):513-520.
14. Duan M, Ma X, Fan J, Guo Y, Wang W, Zhao H, et al. National surveys on 15 quality indicators for the total testing process in clinical laboratories of China from 2015 to 2017. *Clin Chem Lab Med*. 2018;57(2):195-203.
15. Li R, Wang T, Gong L, Dong J, Xiao N, Yang X, et al. Enhance the effectiveness of clinical laboratory critical values initiative notification by implementing a closed-loop system: A five-year retrospective observational study. *J Clin Lab Anal*. 2020;34(2):e23038.
16. Chen J, Wu Z, Liu Y, Wang L, Li T, Dong Y, et al. Prevalence, association relation, and dynamic evolution analysis of critical values in health checkup in China: A retrospective study. *Front Public Health*. 2021;9(630356):1-13.
17. Kopcinovic LM, Trifunović J, Pavosevic T, Nikolac N. Croatian survey on critical results reporting. *Biochem Med (Zagreb)*. 2015;25(2):193-202.
18. Dorgalaleh A, Shiravand Y, Dabbagh A, Tabibian S, Hosseini MS, Mansouri Tourghabeh H, et al. Hemostasis critical values among Iranian clinical laboratories "National Survey of 157 Clinical Laboratories". *Int J Lab Hematol*. 2019;41(6):778-781.
19. Saffar H, Abdollahi A, Hosseini AS, Torabi Farsani M, Hajinasrollah G, Mohaghegh P. Necessity of routine repeat testing of critical values in various working shifts. *Iran J Pathol*. 2020;15(3):161-166.
20. Jafari E, Zarnegar F, Kalantari M, Dabiri S, Naghibzadeh-Tahami A. Critical values in laboratory tests of Iranian patients referring to laboratories: A Cross-sectional study in Kerman. *Arch Iran Med*. 2021;24(6):473-477.
21. Lippi G, Giavarina D, Montagnana M, Luca Salvagno G, Cappelletti P, Plebani M, et al. National survey on critical values reporting in a cohort of Italian laboratories. *Clin Chem Lab Med*. 2007;45(10):1411-1413.
22. Piva E, Sciacovelli L, Laposata M, Plebani M. Assessment of critical values policies in Italian institutions: comparison with the US situation. *Clin Chem Lab Med*. 2010;48(4):461-468.
23. Fadhalah T, Mudaf B, Tawalah H, Fouzan WA, Salem G, Alghanim HA, et al. Baseline assessment of staff perception of critical value practices in government hospitals in Kuwait. *BMC Hlth Ser Res*. 2022;22(986):1-17.
24. Orhan B, Budak H, Topkaya BC, Sonmez D, Ozturk-Emre H, Bercik-Inal B. Analysis of prescribed critical values in Istanbul training and research hospital biochemistry laboratory: Should critical values be repeated? *Clin Lab*. 2020;66(9).
25. Arbiol-Roca A, Corral-Comesaña S, Cano-Corres R, Castro-Castro MJ, Dastis-Arias M, Dot-Bach D. Analysis of laboratory critical values at a referral Spanish tertiary university hospital. *Biochem Med (Zagreb)*. 2019;29(1):010704.
26. Schapkaiz E, Mafika Z. Critical value reporting: a survey of 36 clinical laboratories in South Africa. *S Afr Med J*. 2013;104(1):65-67.
27. Young AB, et al. Management of critical- and significant-risk results. Wayne, PA: CLSI GP47. 2015.
28. Gosselin RC, Adcock D, Dorgalaleh A, Favalaro EJ, Lippi G, Pego JM. International Council for Standardization in Hematology recommendations for hemostasis critical values, tests, and reporting. *Semin Thromb Hemost*. 2020;46(4):398-409.
29. Pereira TC, Clayton AC, Tazelaar HD, Liu Y, Leon M, Silverman JF. Critical values in cytology. *Diagnostic Cytopathology*. 2006;34(6):447-451.
30. Onken AM, VanderLaan PA, Rosenbaum MW. Critical values in cytology. *J Am Soc Cytopathol*. 2021;10(4):341-348.
31. Goh ES, Stavropoulos DJ, Adeli K. Defining and reporting on critical values in genetics: A laboratory survey. *J Appl Lab Med*. 2021;6(5):1299-1304.
32. Schiffman RB, Howanitz PJ, Souers RJ. Point-of-care glucose critical values: A Q-Probes study involving 50 health care facilities and 2349 critical results. *Arch Pathol Lab Med*. 2016;140(2):119-124.
33. Stanford Healthcare. Critical values: Virology. March 22, 2022. Accessed April 18, 2023. <https://stanfordlab.com/test-directory/critical-values.html>.
34. Staats PN, Parkash V, Otis CN, Sharma P, Ioffe O, Bracamonte ER. The current state of communication of urgent and significant, unexpected diagnoses in anatomic pathology. *Arch Pathol Lab Med*. 2020;144(9):1067-1074.
35. Dighe AS, Jones JB, Parham S, Lewandrowski KB. Survey of critical value reporting and reduction of false-positive critical value results. *Arch Pathol Lab Med*. 2008;132(10):1666-1671.
36. Kost GJ. The Coronavirus Disease 2019 spatial care path: Home, community, and emergency diagnostic portals. *Diagnostics*. 2022;12(5):1216, 1-16.
37. Kost GJ. Diagnostic strategies for endemic Coronavirus Disease 2019 (COVID-19). *Arch Pathol Lab Med*. 2022;146(1):16-25.
38. Lehman CM, Howanitz PJ, Souers R, Karcher DS. Utility of repeat testing of critical values: A Q-probes analysis of 86 clinical laboratories. *Arch Pathol Lab Med*. 2014;138(6):788-793.
39. Sun SP, Garcia J, Hayden JA. Repeat critical hematology and coagulation values wastes resources, lengthens turnaround time, and delays clinical action. *Am J Clin Path*. 2018;149(3):247-252.
40. Kost GJ. *The Critical Human: A Prospectus*. Davis, California. Knowledge Optimization, 2023.

CUTOFF CONCENTRATIONS FOR DRUG TESTS

This table was put together by MLO staff referring to federal regulations at 49 CFR part 40, Section 40.87 and information from Thermo Fisher. Section 40.87 states that:

- As a laboratory, you must use the cutoff concentrations displayed in the following table for initial and confirmatory drug tests. All cutoff concentrations are expressed in nanograms per milliliter (ng/mL).
- On an initial drug test, you must report a result below the cutoff concentration as negative. If the result is at or above the cutoff concentration, you must conduct a confirmation test.
- On a confirmation drug test, you must report a result below the cutoff concentration as negative and a result at or above the cutoff concentration as confirmed positive.
- You must report quantitative values for morphine or codeine at 15,000 ng/mL or above.¹

The table follows:

INITIAL TEST ANALYTE	COMMON STREET NAME(S)	INITIAL TEST CUTOFF ^A	CONFIRMATORY TEST ANALYTE	CONFIRMATORY TEST CUTOFF CONCENTRATION
Marijuana metabolites (THCA) ^B	Marijuana (Pot)	50 ng/mL ^C	THCA	15 ng/mL.
Cocaine metabolite (Benzoyllecgonine)	Cocaine (Coke)	150 ng/mL ^C	Benzoyllecgonine	100 ng/mL.
Codeine/Morphine	Opiates	2000 ng/mL	Codeine Morphine	2000 ng/mL. 2000 ng/mL.
Hydrocodone/Hydromorphone	Vicodin/Dilaudid	300 ng/mL	Hydrocodone Hydromorphone	100 ng/mL. 100 ng/mL.
Oxycodone/Oxymorphone	OxyContin/Opana	100 ng/mL	Oxycodone Oxymorphone	100 ng/mL. 100 ng/mL.
6-Acetylmorphine	Heroin	10 ng/mL	6-Acetylmorphine	10 ng/mL.
Phencyclidine	PCP (Angel Dust)	25 ng/mL	Phencyclidine	25 ng/mL.
Amphetamine/Methamphetamine	Meth	500 ng/mL	Amphetamine Methamphetamine	250 ng/mL. 250 ng/mL.
MDMA ^D /MDA ^E	Ecstasy	500 ng/mL	MDMA MDA	250 ng/mL. 250 ng/mL.

REFERENCES

- eCFR :: 49 CFR 40.87 -- What are the cutoff concentrations for drug tests? Ecf.gov. Accessed June 1, 2023. <https://www.ecfr.gov/current/title-49/subtitle-A/part-40/subpart-F/section-40.87>.
- SAMHSA guidelines - US. Accessed June 1, 2023. <https://www.thermofisher.com/us/en/home/clinical/diagnostic-testing/clinical-chemistry-drug-toxicology-testing/drugs-abuse-testing/drug-testing-overview/samhsa.html>.
- A. For grouped analytes (i.e., two or more analytes that are in the same drug class and have the same initial test cutoff):
Immunoassay: The test must be calibrated with one analyte from the group identified as the target analyte. The cross-reactivity of the immunoassay to the other analyte(s) within the group must be 80 percent or greater; if not, separate immunoassays must be used for the analytes within the group.
Alternate technology: Either one analyte or all analytes from the group must be used for calibration, depending on the technology. At least one analyte within the group must have a concentration equal to or greater than the initial test cutoff or, alternatively, the sum of the analytes present (i.e., equal to or greater than the laboratory's validated limit of quantification) must be equal to or greater than the initial test cutoff.
- B. An immunoassay must be calibrated with the target analyte, Δ-9-tetrahydrocannabinol-9-carboxylic acid (THCA).
- C. Alternate technology (THCA and Benzoyllecgonine): When using an alternate technology initial test for the specific target analytes of THCA and Benzoyllecgonine, the laboratory must use the same cutoff for the initial and confirmatory tests (i.e., 15 ng/mL for THCA and 100ng/mL for Benzoyllecgonine).
- D. Methylenedioxymethamphetamine (MDMA).
- E. Methylenedioxyamphetamine (MDA).

**ACHIEVE A CLEANER CLEAN
WITH LESS FRUSTRATION**

20136

19181

Health Care Logistics[®] INC.
GoHCL.com • 1.800.848.1633

**FREE MAGAZINE
SUBSCRIPTION!**
12 Issues

MAGAZINE
mlo-online.com/subscribe

WEBSITE
mlo-online.com

NEWSLETTERS
Sign up for monthly updates

FACEBOOK
[MLOMedicalLaboratoryObserver](https://www.facebook.com/MLOMedicalLaboratoryObserver)

TWITTER
[@MedicalLabMLO](https://twitter.com/MedicalLabMLO)

LINKEDIN
[medical-laboratory-observer](https://www.linkedin.com/company/medical-laboratory-observer)

Scan Here to Subscribe to our FREE Magazine

SPECIMEN	TEST	CONVENTIONAL UNITS	CONVERSION FACTOR (MULTIPLY BY)	SI UNITS
S	Albumin*	3.9-5.1 g/dL	10	39-51 g/L
B	Base excess (men)	-3.3 to +1.2 mmol/L	1	-3.3 to +1.2 mmol/L
B	Base excess (women)	-2.4 to +2.3 mmol/L	1	-2.4 to +2.3 mmol/L
P	Bicarbonate	21-29 mmol/L	1	21-29 mmol/L
S	Bilirubin, conjugated*	<0.3 mg/dL	17.1	<5 µmol/L
S	Bilirubin, total*	0.1-1.2 mg/dL	17.1	2.0-19.9 µmol/L
S/P	Calcium, total	9-10.4 mg/dL	0.25	2.24-2.6 mmol/L
B	CO ₂ content (venous)	22-26 mEq/L	1	22-26 mmol/L
S/P	Chloride*	98-107 mEq/L	1	98-107 mmol/L
S	Cholesterol (NCEP recommendation)	140-200 mg/dL	0.0259	3.6-5.2 mmol/L
S	Cortisol (a.m., total)*	5-23 µg/dL	27.6	138-635 nmol/L
S	Creatinine (Jaffe, men)*	0.9-1.3 mg/dL	88.4	80-115 µmol/L
S	Creatinine (Jaffe, women)*	0.6-1.1 mg/dL	88.4	53-97 µmol/L
S	Ferritin (men)*	39-715 ng/mL	1	39-715 µg/L
S	Ferritin (women)*	6-362 ng/mL	1	6-362 µg/L
P	Fibrinogen	200-400 mg/dL	0.01	2-4 g/L
S	Folate	9.5-39.0 ng/mL	2.265	21.5-88.4 nmol/L
S	Glucose, fasting*	74-100 mg/dL	0.0555	4.1-5.6 mmol/L
S	Haptoglobin*	30-200 mg/dL	0.01	0.3-2.0 g/L
B	Hematocrit (men)*	40.0-52.0 %	0.01	0.40-0.52 Vol fraction
B	Hematocrit (women)*	35.0-47.0 %	0.01	0.35-0.47 Vol fraction
B	Hemoglobin (men)*	14-18 g/dL	10	140-180 g/L
B	Hemoglobin (women)*	12-16 g/dL	10	120-160 g/L
S	Iron, total	20-168 µg/dL	0.179	3.5-30.0 µmol/L
S	Iron binding capacity	250-400 µg/dL	0.179	44.8-71.6 µmol/L
B	Lactate (at bed rest)	5-12 mg/dL	0.111	0.56-1.39 mmol/L
B	Lead	<25 µg/dL	0.048	<1.21 µmol/L
S	Magnesium (Atomic Absorption)	1.6-2.6 mg/dL	0.4114	0.66-1.07 mmol/L
B	MCH (RBC index)*	28.0-32.0 pg/cell	1	28.0-32.0 pg/cell
B	MCHC (RBC index)*	32.0-36.0 g/dL	10	320-360 g/L
B	MCV (RBC index)*	83.0-95.0 fL	1	83.0-95.0 fL
S	Osmolality	280-295 mOsm/kg	1	280-295 mmol/kg
B	pCO ₂ (arterial) (men)	35-48 mm Hg	0.133	4.7-6.4 kPa
B	pCO ₂ (arterial) (women)	32-45 mm Hg	0.133	4.3-6.0 kPa
B	pH (arterial)*	7.35-7.45	1	7.35-7.45
S/P	Phosphate (as P)*	2.8-4.8 mg/dL	0.323	0.89-1.54 mmol/L
B	pO ₂ (arterial)	83-108 mm Hg	0.133	11.0-14.4 kPa
B	Platelet count	150-450 10 ³ /mm ³	1	150-450 10 ⁹ /L
S	Potassium	3.8-4.9 mEq/L	1	3.8-4.9 mmol/L
S	Protein, total (recumbent)	6.0-7.8 g/dL	10	60-78 g/L
B	RBC count (men)*	4.5-5.9 10 ⁶ /mm ³	1	4.5-5.9 10 ¹² /L
B	RBC count (women)*	4.5-5.1 10 ⁶ /mm ³	1	4.5-5.1 10 ¹² /L
S	Sodium	136-145 mEq/L	1	136-145 mmol/L
S	Thyroxine, free*	0.8-2.7 ng/dL	12.9	10.3-34.7 pmol/L
S	Thyroxine (T ₄), total (men)*	4.6-10.5 µg/dL	12.9	59-135 nmol/L
S	Thyroxine (T ₄), total (women)*	5.5-11 µg/dL	12.9	65-138 nmol/L
S	Triglyceride (NCEP recommendation)	10-150 mg/dL	0.0113	0.11-1.7 mmol/L
S	Urea nitrogen (BUN)*	8-24 mg/dL	0.357	2.8-8.6 mmol/L
S	Uric acid (men)*	218-459 µmol/L	59.48	0.22-0.46 mmol/L
S	Uric acid (women)*	147-366 µmol/L	59.48	0.15-0.37 mmol/L
S	Vitamin B12 (WHO Recommendation)	>201 pg/mL	0.733	>147 pmol/L
S	Vitamin D (25-OH)	10-65 ng/mL	2.50	25-162 nmol/L
B	WBC count	4-11 10 ³ /mm ³	1	4-11 10 ⁹ /L
S	Zinc	80-120 µg/dL	0.153	12-18 µmol/L

Specimens: B, whole blood; P, plasma; S, serum. Reference intervals depend on test method and the demographics of the normal population used.

*Adult intervals (18Y-60Y). Age specific ranges apply for pediatric and/or geriatric populations.


Source: Burtis CA, Bruns DE. *Tietz Fundamentals of Clinical Chemistry and Molecular Diagnostics*. 7th ed. St. Louis, MO; Elsevier; 2015 and Rifai, N, Horvath AR, Wittwer, CT. *Tietz Textbook of Clinical Chemistry and Molecular Diagnostics*, 6th ed. St. Louis, MO; Elsevier; 2018 McPherson RA, Pincus MR. *Henry's Clinical Diagnosis and Management by Laboratory Methods*. 22nd ed. Philadelphia, PA: Elsevier Saunders; 22nd ed; 2011. Revised 2023 by S.T. Campbell, PhD, DABCC, FAACC, Director of Chemistry, Montefiore Medical Center, Bronx, NY.

CRITICAL VALUES FOR THERAPEUTIC DRUG LEVELS

The concept of critical values for drug levels was originally developed by the late Daniel M. Baer, MD, and first published in the April 1982 issue of *MLO*. This table lists the critical values for common therapeutic drugs updated in 2023 by Rajasri Chandra, MS, MBA.

DRUG	INDICATION	THERAPEUTIC RANGE	CRITICAL VALUE	COMMENTS
Acetaminophen	Analgesic	5-20 µg/mL	>200 µg/mL *drawn 4 hours after ingestion	*Determination if a concentration is toxic is dependent upon when it is drawn in relation to the time of ingestion of the dose. Multiple serum concentrations will be needed to monitor improvement and removal of drug.
Amikacin	Antimicrobial	Peak: 15-30 µg/mL Trough: 4-8 µg/mL	>10 µg/mL	Peak: 30 minutes after end of infusion. Trough: before next dose. Conventional dosing protocol.
Amiodarone	Antiarrhythmic	0.5-2 µg/mL	>2.5 µg/mL	Trough concentration. Serum amiodarone levels >2.5 µg/mL had a positive predictive value of 76% for adverse drug events.
Amitriptyline	Antidepressant/ analgesic (neuropathic pain)	125-250 ng/mL	>500 ng/mL	Trough concentration. Life threatening cardiac toxicity and/or seizures with concentration >1000 ng/mL.
Busulfan (IV)	Anti-leukemic, Hematopoietic cell transplantation conditioning	900-1350 µmol/min	>1500 µmol/min	Area Under the Curve (AUC) calculations based on post-infusion sampling and dosing protocols vary by institution.
Carbamazepine	Antiepileptic/ mood stabilizer	4-12 µg/mL	>20 µg/mL	Trough concentrations. Correlate serum concentration with clinical presentation.
Cyclosporine	Immunosuppressant	100-400 ng/mL	>500 ng/mL	Specific concentration goal dependent upon clinical situation. For concentrations drawn with intravenous therapy, blood should be drawn from site other than that where drug is infusing. (Cyclosporine adheres to plastic.) TDM levels are dependent on transplant type. Blood concentrations can be method (immunoassay or mass spectrometry) dependent.
Digoxin	Inotrope, AV node blocker	0.5-2.0 ng/mL*	>2.5 ng/mL	Samples should be drawn >8 hours after last dose. *Concentrations >1.5 ng/mL may be associated with higher mortality.
Doxepin	Antidepressant	110-250 ng/mL	>500 ng/mL	Trough concentration.
Ethosuximide	Antiepileptic	40-100 µg/mL	>200 µg/mL	Trough concentration.
Everolimus	Immunosuppressant	3-8 ng/mL	>15 ng/mL	Trough concentration. Varies by transplant protocol.
Flecainide	Antiarrhythmic	0.2-1.0 µg/mL	>1.0 µg/mL	Midpoint or trough concentration. Monitoring recommended when given concurrently with medications that may decrease metabolism (increase concentrations).
Fluconazole	Antifungal	4.0-20.0 µg/mL	None established	Limited TDM utility except in patients receiving hemodialysis.
Flucytosine	Antifungal	25-50 µg/mL	>100-200 µg/mL	Concentration should be a peak drawn 2 hours post dose.
Gentamicin	Antimicrobial	Peak: 5-10 µg/mL Trough: <2 µg/mL	Peak: >12 µg/mL Trough: >2 µg/mL	Peak: 1 hour after infusion. Trough: before next dose. Conventional dosing protocol.
Hydroxyl itraconazole	Antifungal	Not established	None established	Active metabolite of itraconazole.
Imipramine	Antidepressant	>180-240 ng/mL	>500 ng/mL	Concentration = imipramine + desipramine (metabolite).
Itraconazole	Antifungal	>0.5 µg/mL (localized) >1.0 µg/mL (systemic)	None established	Large PK variability. Should be measured within 5-7 after initiation of therapy.
Lamotrigine	Antiepileptic/mood stabilizer	1-15 µg/mL	>20 µg/mL	Trough concentration. High concentrations generally associated with increased somnolence/confusion.
Lidocaine	Antiarrhythmic	1.5-5 µg/mL	>6 µg/mL	Concentration can be drawn at any point (from separate IV line).
Lithium	Mood stabilizer	Acute: 1-1.6 mmol/L Chronic: 0.6-1.2 mmol/L	>2.0 mmol/L >5 mmol/L potentially fatal	Serum concentrations may increase in presence of hyponatremia. Concentration: 12 hours after dose.
Nortriptyline	Antidepressant/ analgesic (neuropathic pain)	50-150 ng/mL	>500 ng/mL	Trough concentration.
Phenobarbital	Antiepileptic	15-40 µg/mL	>60 µg/mL	Trough concentration. Do not collect before steady state achieved.
Phenytoin	Antiepileptic	10-20 µg/mL	>20 µg/mL	Trough concentrations. Toxic >20 µg/mL (lateral nystagmus), >40 µg/mL (decreased mentation). Toxicity may occur at lower concentrations in presence of hypoalbuminemia. Consider free phenytoin.
Posaconazole	Antifungal	>0.7 µg/mL	None established	Should be measured within 7 days of initiation therapy.
Primidone	Antiepileptic	5-12 µg/mL	>15 µg/mL	Metabolized to phenobarbital.
Procainamide (PA) (metabolite: NAPA)	Antiarrhythmic	PA: 4-8 µg/mL NAPA: 10-20 µg/mL	>10 µg/mL >40 µg/mL	Mid-point or trough concentration. Procainamide monitoring is particularly important in patients who might be fast acetylators (60% to 70% of northern Europeans, and 50% of black and white Americans) and in patients with renal impairment. Procainamide and N-acetylprocainamide levels should always be measured on the same sample.
Protriptyline	Antidepressant	50-170 ng/mL	>500 ng/mL	Trough concentration.
Quinidine	Antiarrhythmic	2-5 µg/mL	>6 µg/mL	Midpoint or trough concentration.
Salicylate	Analgesic, antipyresis Anti-inflammatory	20-100 µg/mL 100-200 µg/mL	Vertigo, tinnitus 150-300 µg/mL Nausea, vomiting, hyper-ventilation 250-400 µg/mL Toxicity >500 µg/mL	Serum concentration should be used in conjunction with clinical presentation to make decision on therapy. Multiple serum concentrations will be necessary to monitor improvement and removal of drug.
Sirolimus	Immunosuppressant	4-20 ng/mL	>25 µg/mL	Trough concentration. Whole blood samples. Therapeutic levels can be lower when used in combination with other immunosuppressants. Blood concentrations can be method (immunoassay or mass spectrometry) dependent. Therapeutic levels depend on type of transplant, time post transplant, and other concomitant drug therapy.
Tacrolimus	Immunosuppressant	5-20 ng/mL	>25 ng/mL	Whole blood samples collected as trough. Therapeutic levels can be lower when used in combination with other immunosuppressants. Bias may be present between immunoassay and LC/MS methods.
Theophylline	Bronchodilator	10-20 µg/mL	>25 µg/mL	Pulmonary literature suggest that concentrations 5-15 mg/L may be as efficacious with less toxicity. Trough concentration dependent upon drug formulation.
Tobramycin	Antibacterial	Peak: 4-8 µg/mL Trough: <1.0 µg/mL	>12 µg/mL >2 µg/mL	Peak: 1 hour after end of infusion. Trough: before next dose. Conventional dosing protocol.
Valproic acid	Antiepileptic/mood stabilizer	50-125 µg/mL	>200 µg/mL	Toxicity may occur at lower concentrations in presence of hypoalbuminemia. Consider free valproic acid. Trough concentration preferred.
Vancomycin	Antimicrobial	Trough concentrations: General: 5-15 µg/mL Pneumonia: 15-20 µg/mL	Trough: >30 µg/mL	Monitoring of peaks no longer recommended. Goal trough concentration dependent upon indication. Trough: before next dose.
Voriconazole	Antifungal	1.0-5.5 µg/mL	>6 µg/mL	Should be measured within 7 days of initiation therapy.

Ranges are approximate and may vary with laboratory and/or assay. Proper interpretation of therapeutic drug concentrations requires that the specimen be drawn at an appropriate time in relation to drug administration.

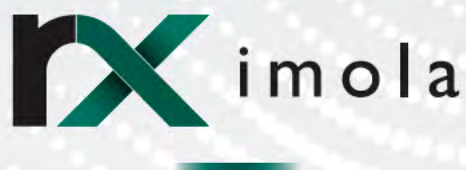

Rapid, Comprehensive Clinical Chemistry Testing

Delivering consistent high-quality results, the RX imola is ideal for medium to high throughput laboratories. Performing 400 photometric tests and 560 tests per hour including ISE, the imola comprises the RX series world leading test menu excelling in performance and functionality.

Why choose the RX imola?

- Routine & Specialized testing can be consolidated onto the RX Imola including Cystatin C, Bilirubin, Lp(a) & sTfR testing.
- Automatic sample re-run and diffraction grating ensures higher accuracy and better precision.
- Direct, on-board, HbA1c testing capabilities provide many time saving benefits in the laboratory.
- Dual reagent probes with liquid level sensors and bubble detection minimize carry over.
- The continuous STAT loading hatch allows emergency samples to be analyzed quickly and easily at any time.



CLR[®]

CLINICAL LABORATORY REFERENCE

PRODUCT INFORMATION

The following section includes company descriptions with their essential laboratory products and contact information for ordering and pricing.

Also available online at:
CLR-online.com

PRODUCT INFORMATION



ADVANCED DATA
SYSTEMS RCM

ADVANCED DATA SYSTEMS CORPORATION

15 Prospect St
Paramus, NJ 07652

ORDERING/PRICING/PRODUCT INFORMATION

TOLL FREE: (800) 899-4237
PHONE: (800) 899-4237
FAX: (201) 497-8023
WEBSITE: www.adsc.com
EMAIL: info@adsc.com

Ideal for claims and billing for any laboratory specialty, MedicsRCM supports a nearly 100% success rate on 1st attempt clearinghouse claims while working to ensure obtaining every dollar for every test. Payer-based verification of benefits alerts. In and out of network alerts for the No Surprises Act. Staffing issues alleviated with our +300 person outsourced workforce. Patients pay online and through their devices and call us with statement questions. Sales activity reports and an optional self serve sales portal. Financial/management reports, KPIs, analytics. LIS integrations. The same system we use, MedicsPremier from ADS, is available if in-laboratory automation is preferred on your server or our cloud.



BD

BD BIOSCIENCES

2350 Qume Dr.
San Jose, CA 95131

ORDERING/PRICING/PRODUCT INFORMATION

PHONE: 855.236.2772
WEBSITE: bdbiosciences.com

BD is a leading global medical technology company that delivers innovative solutions from discovery to diagnosis, continually advancing science and clinical outcomes across infectious disease and cancer. Offerings include complete flow cytometry solutions for the clinical diagnostic laboratory; immunology research solutions, including flow cytometry and multiomics tools; lab automation and informatics solutions; and differentiated reagents and assays.

BD provides a comprehensive portfolio of IVD assays for clinical flow labs, solutions for automated sample preparation, and

clinical analyzers with up to 12 colors with high sensitivity and improved resolution:

Automating the Daily Workflow Improves Productivity and Efficiency

Processing flow cytometry samples requires significant manual labor and results can be impacted by human errors. A workflow using the BD FACSDuet™ Sample Preparation System integrated with the BD FACSLytic™ Flow Cytometer can reduce sample processing time by 20%*, error prone steps by 99%, hands-on time by 75%*, and reduce staff training time.

Clinical Reagents: A complete spectrum with a balance of flexibility and standardization

Highly consistent and unique panel building blocks available in clinical formats (ASR, CE IVD) as well as pre-formulated custom liquid and dried cocktails for efficiency and consistency and fully uniform IVD kits for high level of standardization and compliance.

*Results may vary depending on user factors, including protocol and assay being performed. The results are based on an efficiency study completed by Barts Health NHS Foundation Trust, Royal London Hospital, UK. Automated vs manual preparation when 2 people are dedicated to the task.



BIO-RAD LABORATORIES

4000 Alfred Nobel Dr.
Hercules, CA 94547

ORDERING/PRICING/PRODUCT INFORMATION

TOLL FREE: (800) 224-6723
PHONE: (800) 224-6723
FAX: (510) 741-6373
WEBSITE: www.bio-rad.com/diagnostics

When it comes to quality control, Bio-Rad leads the way for clinical laboratories with a portfolio of independent quality controls and data management software solutions to facilitate effective quality testing, analysis and compliance for hundreds of the most widely used diagnostic instruments. Bio-Rad's comprehensive menu of QC products includes analytes for immunoassay, TDM, chemistry, cardiac assessment, immunology, hematology, diabetes, molecular infectious disease, serological infectious disease and more.

Additional Products & Services Provided:

- Blood Bank Testing
- Clinical Microbiology
- Diabetes and Hemoglobinopathy Testing
- HIV Testing
- Automation Platforms: BioPlex® 2200 System, EVOUS™ microplate system.

- Infectious Disease and Autoimmune Assays
- IntelliQ
- Serum Indices



CAROLINA LIQUID CHEMISTRIES CORP.

313 Gallimore Dairy Rd
Greensboro, NC 27409

ORDERING/PRICING/PRODUCT INFORMATION

PHONE: (877) 722-8910
FAX: (336) 722-8915
WEBSITE: www.carolinachemistries.com
EMAIL: contactsales@carolinachemistries.com

CLC offers high quality, low-cost chemistry analyzers for laboratories of all sizes along with a menu of over 80 different chemistry reagents for use on a variety of instruments. The CLC floor models consist of the CLC800, CLC1600, and the ultra high-speed CLC6410. CLC also offers two benchtop chemistry analyzers: The EasyRA® performs urine drug screens and general chemistry tests, while the DZ-Lite™ c270 offers a large menu of special and general chemistries. Both are excellent for urgent care and oncology practices. The EasyRA and CLC's new RYAN™ point-of-care fentanyl analyzer are also good solutions for qualitative fentanyl testing. CLC continues to sell bar-coded reagents for use on Olympus AU instruments along with affordable service contracts. CLC is Veteran owned and ISO certified.



CLINICAL SOFTWARE SOLUTIONS

PO Box 529
Queen Creek, AZ 85142

ORDERING/PRICING/PRODUCT INFORMATION

TOLL FREE: (800) 570-0474
PHONE: (480) 888-9447 Ext: 102
FAX: (480) 888-9446
WEBSITE: www.clin1mobile.net
EMAIL: info@clin1.com

Clinical Software Solutions has provided software products and services for healthcare facilities for over 30 years. CLIN1 Laboratory and CLIN1 Web LIS offer affordable information management solutions for

any type or size of laboratory. Intuitive user interface with many built-in features including reporting, QC, QA and flexible charge capture help accomplish daily tasks with ease and efficiency. CLIN1 has unique solutions for Toxicology Result Reviewing, Client Services, robust interfacing and Web Portal reporting to increase productivity and improve workflow. Enhance your laboratory environment with healthcare software that delivers value. Workflow Customizations are also available.



COLLEGE of AMERICAN PATHOLOGISTS

COLLEGE OF AMERICAN PATHOLOGISTS

325 Waukegan Rd
Northfield, IL 60093-2750

ORDERING/PRICING/PRODUCT INFORMATION

TOLL FREE: (800) 323-4040
PHONE: (847) 832-7000
WEBSITE: www.cap.org
EMAIL: contactcenter@cap.org

The College of American Pathologists (CAP) partners with laboratories worldwide to elevate the quality of laboratory medicine with best-in-class solutions designed to drive operational excellence, achieve diagnostic confidence, and ensure the best patient care. Laboratories rely on the CAP for:

- Laboratory accreditation programs to improve quality and reduce risk
- 700+ PT/EQA programs to ensure accurate results across all disciplines, many with complimentary enrollment for online continuing education
- Protocols and guidelines to improve cancer reporting and clinical decision making
- Quality improvement tools to improve processes and strengthen staff competency

Built on a foundation of pathologist expertise, the CAP supports customers in rapidly responding to changes in laboratory medicine. Empowering laboratories to achieve higher quality standards, the CAP offers its core Laboratory Accreditation Program and specialty accreditation programs for biorepository, reproductive, and forensic drug testing. For laboratories pursuing ISO 15189 recognition, the CAP 15189 program aligns with the updated ISO standards.

See our ad on the Inside Front Cover



COMPUGROUP MEDICAL US

10065 Red Run Blvd., Ste. 150
Owings Mills, MD 21117

ORDERING/PRICING/PRODUCT INFORMATION

PHONE: (800) 359-0911 Ext: Opt. 3

WEBSITE: www.cgm.com/us

EMAIL: sales.lab.us@cgm.com

CompuGroup Medical is the leading vendor for laboratory information systems in the U.S. and offers four renowned platforms: CGM LABDAQ, CGM MEDICUS, CGM SCHUYLAB, and CGM AP EASY.

Designed to increase efficiency and maximize your reimbursements, CGM offers scalable solutions that will grow with your lab. For independent labs, an optional web portal allows providers without EMR connectivity to place orders and receive results.

Similarly, CGM AP EASY simplifies the anatomic pathology workflow for AP testing labs.

CGM will guide you to the software and configuration that makes the most sense for your unique needs, workflow, and specialty.



DENLINE UNIFORMS

301 Oak Street
Quincy, IL 62301

ORDERING/PRICING/PRODUCT INFORMATION

TOLL FREE: (800)-DENLINE (336-5463)

PHONE: (217) 228-9272

WEBSITE: <https://www.denlineuniforms.com/>

EMAIL: customerservice@denlineuniforms.com

Industry Leading Producer of Fluid Resistant PPE For Clinical Laboratory Personnel with exposure to Blood and OPIM. All Apparel Tested AAMI PB70 Level 2 after completing 225 Commercial Washes; Durable 200+ Wash Rated. Styles assist with compliance to OSHA Directive(s) 29 CFR1910. Clean Room ISO 5/Class 100 Fabric, Anti-static, Stain Resistant to Bleach, ALL Aqueous Liquids.

10 Popular Unisex and Ladies Styles, 4 colors, sizes XS to 5XL. NEW UltraLite Styles provide Air Permeability/Comfort of

a cotton/polyester lab coat. Coats laundered using CDC/HICPAC Recommended Sanitizing Guidelines.

DenLine stocks 90,000 coats for prompt shipments. 100% US Manufactured Fabric, Sewn in Near-Shore US, WRAP Certified Production Plants.

Supplier of DenLine AAMI Level 2 Disposable Lab Coats & Jackets; Industry Leading Air Permeability for wearer comfort, antistatic, latex free, sizes XS-5XL. Samples Available.



DIASORIN MOLECULAR

11331 Valley View Street
Cypress, CA 90630

ORDERING/PRICING/PRODUCT INFORMATION

PHONE: (562) 240-6500

FAX: (714) 243-4703

WEBSITE: www.molecular.diasorin.com

EMAIL: CS.molecular@diasorin.com

DiaSorin Molecular manufactures and distributes molecular diagnostic products worldwide, helping laboratories streamline workflow and improve patient management. Our expanding LIAISON® MDX menu now includes the only FDA-cleared assay for both saliva swab and urine samples for the detection of congenital cytomegalovirus (cCMV). Simplexa® Congenital CMV Direct will support your youngest patients with fast, accurate, and actionable diagnosis of cCMV in babies up to 21 days old. Through our molecular diagnostic solutions, we seek to better patient management with fast, reliable, and clinically actionable assay results. Also visit Luminex, a DiaSorin Company, for more laboratory testing solutions.

See our ad on the Inside Back Cover



GLOBE SCIENTIFIC

400 Corporate Drive
Mahwah, NJ 07430

ORDERING/PRICING/PRODUCT INFORMATION

TOLL FREE: (800) 394-4562

PHONE: (201) 599-1400

FAX: (201) 599-1406

WEBSITE: www.globescientific.com/

EMAIL: mail@globescientific.com

Elevating standards in labware since 1983, Globe Scientific is a leading provider of premium quality laboratory plasticware, glassware, and benchtop equipment. We cater to a global network of dealers and laboratory professionals across diverse sectors, including clinical, research, education, veterinary, and industrial markets. Boasting an extensive inventory of over 4500 items available for immediate shipment, we ensure prompt and reliable delivery.

At Globe Scientific, our mission is to continually broaden and enhance our product portfolio, driven by a family-oriented culture that places paramount importance on delivering 5-star service and unparalleled value to our partners. Our unwavering commitment to being the most responsive company in the industry fuels our drive to consistently exceed customer expectations.

Supporting our pursuit of excellence, Globe Scientific proudly holds ISO certification, a testament to our dedication to maintaining rigorous quality control protocols and adhering to industry standards. To explore our world-class offerings and learn more about our commitment to exceptional products and service, visit us at www.globescientific.com.



HARDY DIAGNOSTICS

1430 W McCoy Ln
Santa Maria, CA 93455

ORDERING/PRICING/PRODUCT INFORMATION

PHONE: (800) 266-2222

WEBSITE: hardydiagnostics.com

EMAIL: marketing@hardydiagnostics.com

Hardy Diagnostics is an FDA-licensed manufacturer of medical devices for microbiological testing with an ISO 13485 certified Quality Management System. The company manufactures and distributes thousands of products for the culture and identification of bacteria and fungi. Hardy Diagnostics is headquartered in Santa Maria, California, and services over 10,000 laboratories across the nation. Hardy Diagnostics exports products to over 80 foreign distributors and maintains nine distribution centers nationwide. Hardy Diagnostics' mission is to produce and distribute the finest products for the detection of microorganisms and partner with its laboratory customers to diagnose and prevent disease. For more information on products and services, visit www.HardyDiagnostics.com.



HEALTH CARE LOGISTICS

P.O. Box 25
Circleville, OH 43113

ORDERING/PRICING/PRODUCT INFORMATION

TOLL FREE: (800) 848-1633
PHONE: (740) 477-3755
FAX: (800) 447-2923
WEBSITE: www.gohcl.com/
EMAIL: hcl@GoHCL.com

Health Care Logistics® is an industry leader specializing in manufacturing, packaging and distributing unique and hard-to-find products. Top inventory lines include Lab Equipment, Biohazard Supplies and Specimen Handling Materials. The company's commitment to customer satisfaction sets it apart with services such as free samples, small package quantities and same-day shipping on most items. HCL has spent more than 40 years tailoring its products and services to the needs of its customers and will scout non-inventory items at no extra charge or manufacture custom solutions based on individual requests.

See our ad on page 8



KRONUS, INC.

170 South Seneca Springs Way, Suite 105
Star, ID 83669

ORDERING/PRICING/PRODUCT INFORMATION

TOLL FREE: (800) 457-6687
PHONE: (208) 377-4800
FAX: (208) 377-4900
WEBSITE: www.kronus.com
EMAIL: kronus@kronus.com

For over 35 years, KRONUS® has provided specialized immunoassay test kits to medical professionals at the world's most respected laboratory facilities. Please contact us directly with any questions, requests or comments that you may have. We welcome the opportunity to learn how KRONUS can be of assistance in meeting your laboratory's immunoassay test kit needs.

ISLET CELL AUTOIMMUNITY

ZINC TRANSPORTER 8 AUTOANTIBODY (ZnT8Ab)

- ELISA Kit, 96 Wells

GLUTAMIC ACID DECARBOXYLASE AUTO-ANTIBODY (GADAb)

- ELISA Kit, 96 Wells

IA-2 AUTOANTIBODY (IA-2Ab)

- ELISA Kit, 96 Wells

INSULIN AUTOANTIBODY (IAA)

- Radioimmunoassay Kit, 100 Tubes

NEUROIMMUNOLOGY

AQUAPORIN-4 AUTOANTIBODY (AQP4Ab)

- ELISA Kit, 96 Wells

ACETYLCHOLINE RECEPTOR AUTOANTIBODY (AChRab): BINDING; BLOCKING

- Radioimmunoassay Kits, 100 Tubes

VOLTAGE-GATED CALCIUM CHANNEL AUTOANTIBODY (VGCCAb): P/Q-TYPE

- Radioimmunoassay Kits, 12 and 24 Tubes

THYROID AUTOIMMUNITY

TSH RECEPTOR AUTOANTIBODY (TRAb)

- ELISA Kit, 96 Wells
- Coated-Tube (CT) Radioimmunoassay Kits, 60 and 120 Tubes

ADRENAL AUTOIMMUNITY

STEROID 21-HYDROXYLASE AUTOANTIBODY (21-OHAb)

- ELISA Kit, 96 Wells

FOR RESEARCH USE

3 SCREEN ISLET CELL (GAD/IA-2/ZnT8) AUTOANTIBODY SCREEN

- ELISA Kit, 96 Wells†

MUSCLE SPECIFIC TYROSINE KINASE AUTOANTIBODY (MuSKAb)

- Radioimmunoassay Kit, 25 Tubes†

TITIN AUTOANTIBODY

- ELISA Kit, 48 Wells†

ACETYLCHOLINE RECEPTOR AUTOANTIBODY (AChRab) – BINDING

- ELISA Kit, 96 Wells†

ACETYLCHOLINE RECEPTOR AUTOANTIBODY (AChRab) – GANGLIONIC

- Radioimmunoassay Kit, 25 Tubes†

VOLTAGE-GATED POTASSIUM CHANNEL AUTOANTIBODY (VGKAb)

- Radioimmunoassay Kit, 25 Tubes†

VOLTAGE-GATED CALCIUM CHANNEL AUTOANTIBODY (VGCCAb): N-TYPE

- Radioimmunoassay Kit, 25 Tubes†

AQUAPORIN-4 AUTOANTIBODY (AQP4Ab)

- IFA Kit, 100 Fields†

†For Research Use Only. Not For Use in Diagnostic Procedures.

See our ad on page 3



**Laboratory Growth
& Productivity**

Consulting, Inc.

LGP (LABORATORY GROWTH & PRODUCTIVITY)

PO Box 18
Woodriver, IL 62095

ORDERING/PRICING/PRODUCT INFORMATION

TOLL FREE: (877) 251-9246
PHONE: (618) 251-9246
FAX: (618) 251-9256
WEBSITE: www.LGPconsulting.com
EMAIL: sales@LGPconsulting.com

Is Safety your concern? Then Pluggo Decapper(s) and KapSafe Recapper(s) are your solutions. Both are bench-top, proven, reliable and affordable. Eliminate possible injury during manual decapping and recapping. Serving labs since 2002.

Luminex®

A DiaSorin Company

LUMINEX

12212 Technology Blvd Ste 130
Austin, TX 78727

ORDERING/PRICING/PRODUCT INFORMATION

TOLL FREE: (877) 785 2323
PHONE: (512) 219-8020
FAX: (512) 219 5195
WEBSITE: www.luminexcorp.com
EMAIL: orders@luminexcorp.com

At Luminex, a DiaSorin Company, our mission is to empower labs to obtain reliable, timely, and actionable answers, ultimately advancing health. We serve our customers' needs in diverse markets, including life science research, clinical diagnostics, immunology, and personalized medicine. Our goal is to globally transform healthcare by developing innovative instruments and assays that are clinically reliable, operationally efficient, and economically responsible. Check out our instruments and reagents for your targeted and syndromic molecular testing, and our xMAP® Technology for your specified multiplexing needs. Also, visit DiaSorin Molecular for more molecular diagnostic solutions.



ORCHARD SOFTWARE CORPORATION

701 Congressional Blvd.
Carmel, IN 46032

ORDERING/PRICING/PRODUCT INFORMATION

TOLL FREE: (800) 856-1948
PHONE: (800) 856-1948
WEBSITE: orchardsoft.com
EMAIL: sales@orchardsoft.com

Orchard Software Corporation is a leader in the laboratory information system industry and offers a variety of laboratory system solutions. Orchard serves more than 2,000 laboratories across the country, helping them improve efficiency, reduce errors, and enhance integration. Orchard's cloud-based solutions are installed in physician groups and clinics, hospitals, independent reference labs, student health centers, veterinary labs, public health organizations, universities, and retail facilities. Orchard provides a comprehensive suite of laboratory information systems covering clinical lab, microbiology, pathology, molecular, point-of-care testing, and toxicology.



PROLINE LABORATORY FURNITURE

10 Avco Rd.
Haverhill, MA 01835

ORDERING/PRICING/PRODUCT INFORMATION

PHONE: (800) 739-9067
FAX: (978) 374-4885
WEBSITE: www.1proline.com
EMAIL: Bench@1proline.com

Proline Laboratory Furniture and Tables offers a modular design that allows you to configure laboratory to meet your requirement. Offering multiple work surface choices including but not limited to Epoxy Resin, Stainless Steel and economical Chem-Guard chemical resistant work surfaces. Cabinets offer 90 percent extension slides with 150 pounds capacity. Economical laboratory tables with hanging cabinets and casters offering a truly modular ergonomic design.



PURITAN MEDICAL PRODUCTS

31 School Street, Box 149
Guilford, ME 04443

ORDERING/PRICING/PRODUCT INFORMATION

TOLL FREE: (800) 321-2313
PHONE: (207) 876-3311
WEBSITE: www.puritanmedproducts.com
EMAIL: sales@puritanmedproducts.com

Puritan Medical Products Co., is a 104-year-old, family-owned American company known worldwide as the trusted manufacturer of single-use products for controlled environments, diagnostics, and specimen collection, environmental, forensics, and genetics, medical and microbiology media industries. Puritan is FDA-registered and ISO-certified. Its products help customers collect, analyze, and transport specimens with the utmost confidence in the products' clinical integrity. Puritan manufactures all its patented flock swabs and related products in its US-based manufacturing facilities. For more information about Puritan Medical Products, visit www.puritanmedproducts.com



RANDOX LABORATORIES-US, LTD.

515 Industrial Boulevard
Kearneysville, WV 25430

ORDERING/PRICING/PRODUCT INFORMATION

TOLL FREE: (866) 4 Randox
PHONE: (304) 728-2890
FAX: (304) 728-1890
WEBSITE: www.randox.com
EMAIL: marketing@randox.com

At Randox, our dedication lies in creating groundbreaking diagnostic solutions that enhance global health. Our portfolio includes chemistry, immunoassay, molecular, and POC analyzers, coupled with superior quality first and third-party reagents. As a premier provider of comprehensive quality control solutions, we offer daily quality control, calibration verification, and proficiency

testing to ensure dependable results. Our Biochip Array Technology is a distinct multiplexing approach that features clinical, research, and molecular panels.

Randox is a company that is at the forefront of healthcare innovation, with a strong focus on improving health outcomes worldwide through our cutting-edge products and services. Our commitment to science and dedication to advancing diagnostic testing make Randox the trusted partner for healthcare professionals.

See our ad on page 11



Because every result matters™

SEKISUI DIAGNOSTICS

One Wall Street
Burlington, MA 01803

ORDERING/PRICING/PRODUCT INFORMATION

PHONE: (781) 652-7800
WEBSITE: www.sekisuidiagnostics.com
EMAIL: Questions@Sekisui-Dx.com

For over 40 years, we have supplied innovative medical diagnostics globally to physicians, hospitals, laboratories and alternate testing locations.


We develop, manufacture, and supply billions of tests each year through our commercial networks and partners. Our product lines include:

- Clinical chemistry reagents
- Point-of-care tests
- Pre-analytic systems
- Enzymes and specialty biochemicals

We are part of SEKISUI Medical Group; a global family of four companies in seven countries, employing over 2,000 people from varied backgrounds and cultures.

Our facilities and offices span the globe—from California to Japan— all with the shared mission of providing intelligent solutions to enhance life with science and improve the health of all people.

See our ad on the Back Cover

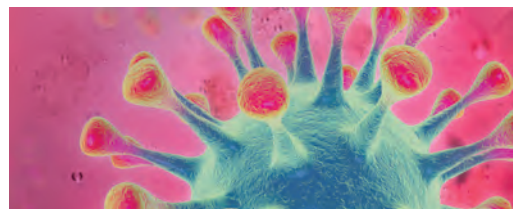

SOFTGENETICS®

Software PowerTools for Genetic Analysis

SOFTGENETICS

100 Oakwood Ave
State College, PA 16803

ORDERING/PRICING/PRODUCT INFORMATION

PHONE: (814) 237-9340 Ext: 202
FAX: (814) 237-9343
WEBSITE: www.softgenetics.com
EMAIL: info@softgenetics.com

SoftGenetics LLC specializes in the development of genetic analysis software **PowerTools** for medical research and forensic applications. Hallmarks of SoftGenetics' tools are advanced technologies providing exceptional accuracy, and sensitivity in an easy-to-use Windows® user interface.

Programs Include:

Mutation Surveyor® Software, for the analysis of capillary sequence data

GeneMarker® Software, for MLPA®, MSI, Fragile X, CF, Aneuploidy, all fragment analysis applications

ChimerMarker®, automated chimera analysis and long-term monitoring software

GeneMarker®MTP, Multiple Template Processor, like having 6 GeneMarker software in one!

NextGENE® Software, analysis of NGS reads, Variants including SNVs, indels and somatic mutations, CNV, Structural Variations including gene fusions, RNA-seq

Geneticist Assistant™ Software, NGS Interpretative Workbench

GeneMarker®HTS Software, analysis of NGS mitochondrial & STR's for forensic applications

MaSTR™, Enlightened Probabilistic Mixture Analysis Software

GeneMarker®HID Software, NDIS approved Expert System for forensic STR Analysis

NextGENE®LR, for analysis of Oxford Nanopore MinION and PacBio long read systems

Visit www.softgenetics.com for technical support and to request program trials!

StaffReady

Clinical Workforce Management Software

STAFFREADY

421 W. Main Ave, Suite 201
Spokane, WA 99201

ORDERING/PRICING/PRODUCT INFORMATION

TOLL FREE: (877) 229-5230
PHONE: (509) 252-3550 Ext: 106
WEBSITE: www.staffready.com
EMAIL: info@staffready.com

Most healthcare leaders face complex demands and regulatory pressures with too few resources. StaffReady's staff scheduling and compliance solutions simplify operations so you can sleep at night.

Since 2002, our goal has been to understand the operational needs of the healthcare market and bring cutting-edge, cloud-based solutions to assist with those challenges.

StaffReady's flexible but robust software solutions centralize schedules, competencies, documents, policies, tasks, alerts and all staff and management communication. Leverage our rich toolset of integrated, web-based quality management modules to streamline operations and keep everyone in your organization operating from the same platform, no matter how large your staff or how many locations you have.

techcyte

TECHCYTE

105 S State Street, #401
Orem, UT 84058

ORDERING/PRICING/PRODUCT INFORMATION

TOLL FREE: (888) 878-3249
PHONE: (801) 735-2494
WEBSITE: techcyte.com
EMAIL: info@techcyte.com

Techcyte is the leading Clinical Pathology AI and workflow platform for liquid and cellular sample types. Techcyte has deep learning image analysis solutions for:

- fecal ova & parasite detection
- cervical cancer screening
- automated blood differentials
- bacterial vaginosis gram stain analysis
- digital manual review of other bacteriology samples
- bladder cancer detection
- and remote pathology for digital cytology and FNA consultations

Techcyte enables labs to increase efficiencies, reduce labor shortages, and improve accuracy with greater consistency. Techcyte's workflow and AI platform is focused on delivering quicker, more accurate results to patients through digital diagnostics.



WSLH Proficiency Testing
WISCONSIN STATE LABORATORY OF HYGIENE
UNIVERSITY OF WISCONSIN-MADISON

WSLH PROFICIENCY TESTING

2601 Agriculture Dr.
Madison, WI 53718

ORDERING/PRICING/PRODUCT INFORMATION

TOLL FREE: (800) 462-5261
PHONE: (608) 224-4318
FAX: (608) 265-1111
WEBSITE: <https://www.wslhpt.org>
EMAIL: ptservice@slh.wisc.edu

YOUR LAB, OUR MISSION

With WSLH Proficiency Testing, you receive reliable, affordable, and easy-to-use laboratory proficiency testing products with rapid access to results. Connect with our dedicated team to request a quote anytime.

We are your proficiency testing provider, backed by a national public health lab and a Big Ten university. Our unique affiliations give you the educational resources and support you need for successful proficiency testing.

WSLH Proficiency Testing, building a solid foundation for clinical laboratory improvement since 1966. WSLH PT is approved by CMS, and accepted by CAP, COLA, and The Joint Commission. Learn more at wslhpt.org.

**Visit CLR online
all year at
www.clr-online.com/**





CLR[®]

CLINICAL LABORATORY REFERENCE

PRODUCT INDEX

The following section includes company tests, equipment, products and services for the clinical lab market.

Also available online at:
CLR-online.com

	Page		Page		Page
#		A			
21-Hydroxylase antibody		Aerobic bacterial culture, general		Amikacin assays (AMIK)	
KRONUS, Inc.....	15	Hardy Diagnostics.....	14	Randex Laboratories-US, Ltd.....	16
21-Hydroxylase autoantibody kits		Alanine aminotransferase assays		Ammonia assays (AMMON)	
KRONUS, Inc.....	15	SEKISUI Diagnostics	16	Randex Laboratories-US, Ltd.....	16
Techcyte	17	Albumin		Ammonia, control	
		Carolina Liquid		Bio-Rad Laboratories	13
		Chemistries Corp.....	13	Randex Laboratories-US, Ltd.....	16
		Randex Laboratories-US, Ltd.....	16	Ammonia, plasma	
		SEKISUI Diagnostics	16	SEKISUI Diagnostics	16
a1-Acid glycoprotein		Albumin, assays		Ammonia, standard	
Randex Laboratories-US, Ltd.....	16	Randex Laboratories-US, Ltd.....	16	Randex Laboratories-US, Ltd.....	16
a1-Acid glycoprotein, immunoassay		Albumin, standard		Amphetamine assays	
Randex Laboratories-US, Ltd.....	16	Randex Laboratories-US, Ltd.....	16	Carolina Liquid	
a1-Antitrypsin assays		Alcohol assays		Chemistries Corp.....	13
Randex Laboratories-US, Ltd.....	16	Carolina Liquid		Randex Laboratories-US, Ltd.....	16
Accreditation services, laboratory		Chemistries Corp.....	13	Amphetamine assays, urine	
College of American		Randex Laboratories-US, Ltd.....	16	Carolina Liquid	
Pathologists	13	Aldolase		Chemistries Corp.....	13
Hardy Diagnostics.....	14	Carolina Liquid		Randex Laboratories-US, Ltd.....	16
Accuracy-based products		Chemistries Corp.....	13	Amphetamine confirmation, urine	
College of American		Randex Laboratories-US, Ltd.....	16	Randex Laboratories-US, Ltd.....	16
Pathologists	13	Alkaline phosphatase (ALP)		Amphetamine screen and confirmation, urine	
WSLH Proficiency Testing	17	Carolina Liquid		Randex Laboratories-US, Ltd.....	16
Acetaminophen assays (ACETA)		Chemistries Corp.....	13	Amylase (AMY)	
SEKISUI Diagnostics	16	SEKISUI Diagnostics	16	Carolina Liquid	
Acetaminophen, serum		Alkaline phosphatase assays		Chemistries Corp.....	13
SEKISUI Diagnostics	16	Randex Laboratories-US, Ltd.....	16	Randex Laboratories-US, Ltd.....	16
Acetylcholine receptor (AChR)-binding antibody		Allergen profile, latex-related		Amylase (AMY), serum	
KRONUS, Inc.....	15	Hardy Diagnostics.....	14	Carolina Liquid	
Acetylcholine receptor antibody kits		Allergy testing products and services		Chemistries Corp.....	13
KRONUS, Inc.....	15	Bio-Rad Laboratories	13	Randex Laboratories-US, Ltd.....	16
Acetylcholine receptor-blocking antibodies, serum		ALP (IFCC)		Amylase, assays	
KRONUS, Inc.....	15	Randex Laboratories-US, Ltd.....	16	Randex Laboratories-US, Ltd.....	16
AchRAb (See Acetylcholine receptor antibody kits)		Alpha-1 acid glycoprotein (AAG)		SEKISUI Diagnostics	16
KRONUS, Inc.....	15	Randex Laboratories-US, Ltd.....	16	Anaerobic and aerobic culture	
Acute leukemia profile		Alpha-1-antitrypsin, serum (AAT)		Hardy Diagnostics.....	14
BD Biosciences.....	12	Randex Laboratories-US, Ltd.....	16	Anaerobic culture	
Adenovirus		Alpha-1-microglobulin		Hardy Diagnostics.....	14
Luminex.....	15	Randex Laboratories-US, Ltd.....	16	Anaerobic culture, aerobic culture and gram's stain	
Adenovirus DNA, qualitative, RT PCR		Alpha-1-microglobulin, urine		Hardy Diagnostics.....	14
DiaSorin Molecular.....	14	Randex Laboratories-US, Ltd.....	16	Techcyte	17
Luminex.....	15	Alpha-fetoprotein		Analyzers	
Adenovirus testing		Randex Laboratories-US, Ltd.....	16	Luminex.....	15
Luminex.....	15	ALT (Alanine aminotransferase)		Randex Laboratories-US, Ltd.....	16
		Carolina Liquid		Analyzers, automated	
		Chemistries Corp.....	13	Carolina Liquid	
		Randex Laboratories-US, Ltd.....	16	Chemistries Corp.....	13
		ALT (IFCC)		Luminex.....	15
		Randex Laboratories-US, Ltd.....	16	Randex Laboratories-US, Ltd.....	16
				Analyzers, benchtop	
				Carolina Liquid	
				Chemistries Corp.....	13
				Luminex.....	15
				Randex Laboratories-US, Ltd.....	16
				Analyzers, benchtop	
				Carolina Liquid	
				Chemistries Corp.....	13
				Luminex.....	15
				Randex Laboratories-US, Ltd.....	16
				Analyzers, benchtop	
				Carolina Liquid	
				Chemistries Corp.....	13
				Luminex.....	15
				Randex Laboratories-US, Ltd.....	16
				Analyzers, benchtop	
				Carolina Liquid	
				Chemistries Corp.....	13
				Luminex.....	15
				Randex Laboratories-US, Ltd.....	16
				Analyzers, benchtop	
				Carolina Liquid	
				Chemistries Corp.....	13
				Luminex.....	15
				Randex Laboratories-US, Ltd.....	16
				Analyzers, benchtop	
				Carolina Liquid	
				Chemistries Corp.....	13
				Luminex.....	15
				Randex Laboratories-US, Ltd.....	16

Page	Page	Page	Page
Antisera (See Reagents, immunologic)	AST (See Aspartate aminotransferase)	Barbiturate screen, urine	Bilirubin, total and direct, serum
Randox Laboratories-US, Ltd. 16	Randox Laboratories-US, Ltd. 16	Randox Laboratories-US, Ltd. 16	Carolina Liquid
Antisera (See Reagents, immunologic), bulk	Autoimmune disease testing	Barbiturates, assays	Chemistries Corp. 13
Randox Laboratories-US, Ltd. 16	KRONUS, Inc. 15	Randox Laboratories-US, Ltd. 16	Billing and accounts receivable
Antisera (See Reagents, immunologic), polyclonal	Automated analyzers	Barbiturates, assays, serum	Advanced Data Systems
Randox Laboratories-US, Ltd. 16	Hardy Diagnostics 14	Randox Laboratories-US, Ltd. 16	Corporation 12
Antisera (See Reagents, immunologic), standard	Randox Laboratories-US, Ltd. 16	Barbiturates, assays, urine	Biochemistry products and services
Randox Laboratories-US, Ltd. 16	Automated hematology	Carolina Liquid	Carolina Liquid
Antistreptolysin O (ASO) antibodies	College of American	Chemistries Corp. 13	Chemistries Corp. 13
Randox Laboratories-US, Ltd. 16	Pathologists 13	Randox Laboratories-US, Ltd. 16	Randox Laboratories-US, Ltd. 16
Antistreptolysin-O assays	Techcyte 17	Barbiturates, blood	Biotin-conjugates
Randox Laboratories-US, Ltd. 16	Automated systems	Randox Laboratories-US, Ltd. 16	BD Biosciences 12
Apo A	Advanced Data Systems	Barbiturates, serum	BK virus DNA, PCR
Randox Laboratories-US, Ltd. 16	Corporation 12	Randox Laboratories-US, Ltd. 16	DiaSorin Molecular 14
Apo B	Clinical Software Solutions 13	Barcode products	Bladder cancer, FISH
Randox Laboratories-US, Ltd. 16	Hardy Diagnostics 14	Luminex 15	Techcyte 17
Apolipoprotein A1 (APO A1)	Randox Laboratories-US, Ltd. 16	B-Cell CD20 Expression	Bladder tumor antigen (BTA) rare
Carolina Liquid	Autopsy	BD Biosciences 12	Techcyte 17
Chemistries Corp. 13	College of American	B-Cell Immunodeficiency Profile	Bladder tumor antigen (BTA)/cytology profile rare
Randox Laboratories-US, Ltd. 16	Pathologists 13	BD Biosciences 12	Techcyte 17
Apolipoprotein assessment	Avidin-streptavidin conjugates	Benchmarking services	Bladder tumor antigen (BTA)/cytology/DNA ploidy analysis profile rare
Randox Laboratories-US, Ltd. 16	BD Biosciences 12	College of American	Techcyte 17
Apolipoprotein B (APO B)	B	Pathologists 13	Bladder tumor antigen (BTA)/cytology/DNA ploidy analysis profile rare
Carolina Liquid	Babesia microti antibody profile	Benzodiazepines, assays, urine	Techcyte 17
Chemistries Corp. 13	DiaSorin Molecular 14	Carolina Liquid	Bladder tumor DNA ploidy analysis profile rare
Randox Laboratories-US, Ltd. 16	Bacterial vaginosis (sialidase) and vaginal yeast (culture)	Chemistries Corp. 13	Techcyte 17
Apolipoproteins	Bacterial vaginosis (sialidase), chlamydia trachomatis, neisseria gonorrhoeae, trichomonas vaginalis (NAA), vaginal yeast culture	Bicarbonate (CO2)	Blood bank
Randox Laboratories-US, Ltd. 16	Techcyte 17	Carolina Liquid	College of American
Apolipoproteins, assays	Bacterial vaginosis (sialidase), Tv (NAA), vaginal yeast (culture)	Chemistries Corp. 13	Pathologists 13
Randox Laboratories-US, Ltd. 16	Techcyte 17	Bile acids, assays	Blood bank, products
Apparel	Bacterial vaginosis (sialidase), Tv (NAA), vaginal yeast (culture)	Randox Laboratories-US, Ltd. 16	Globe Scientific 14
Denline Uniforms 14	Techcyte 17	Bile acids, total	Blood bank, quality control
Arbovirus	Bacterial vaginosis (sialidase), Tv (NAA), vaginal yeast (culture)	Carolina Liquid	Bio-Rad Laboratories 13
DiaSorin Molecular 14	Techcyte 17	Chemistries Corp. 13	College of American
ASO (See Antistreptolysin-O)	Bacterial vaginosis (sialidase), Tv (NAA), vaginal yeast (culture)	Randox Laboratories-US, Ltd. 16	Pathologists 13
Randox Laboratories-US, Ltd. 16	Techcyte 17	Bilirubin assays	Blood bank, reagents
Aspartate aminotransferase (AST)	Bacterial vaginosis, sialidase activity	Randox Laboratories-US, Ltd. 16	Bio-Rad Laboratories 13
Carolina Liquid	Techcyte 17	Bilirubin assays, controls	Blood culture, routine
Chemistries Corp. 13	Bacteriology (Microbiology)	Randox Laboratories-US, Ltd. 16	College of American
SEKISUI Diagnostics 16	College of American	Bilirubin assays, direct	Pathologists 13
Assay development service	Pathologists 13	Carolina Liquid	Blood gas
Randox Laboratories-US, Ltd. 16	Puritan Medical Products 16	Chemistries Corp. 13	College of American
Assay matrix development service	Techcyte 17	Randox Laboratories-US, Ltd. 16	Pathologists 13
Randox Laboratories-US, Ltd. 16	Bacteriology test reports	SEKISUI Diagnostics 16	Randox Laboratories-US, Ltd. 16
AST	Techcyte 17	Bilirubin assays, standard	Blood glucose (See Glucose, assays)
Carolina Liquid	Barbiturate confirmation, urine	College of American	Carolina Liquid
Chemistries Corp. 13	Randox Laboratories-US, Ltd. 16	Pathologists 13	Chemistries Corp. 13
AST (IFCC)	Barbiturate confirmation, urine	Randox Laboratories-US, Ltd. 16	Randox Laboratories-US, Ltd. 16
Randox Laboratories-US, Ltd. 16	Randox Laboratories-US, Ltd. 16	SEKISUI Diagnostics 16	Blood screening

Page	Page	Page	Page
Blood screening, reagent kits Randox Laboratories-US, Ltd. 16	C	Candida Luminex..... 15	Cell-adhesion molecule reagents Randox Laboratories-US, Ltd. 16
Blood screening, specialty BD Biosciences..... 12	CO2 (Carbon Dioxide) Carolina Liquid Chemistries Corp. 13	Candida, testing Luminex..... 15	Certification College of American Pathologists 13
Blood, analysis Techcyte 17	CA-125 Randox Laboratories-US, Ltd. 16	Cannabinoid (marijuana) confirmation, urine Randox Laboratories-US, Ltd. 16	Chemistry College of American Pathologists 13 Randox Laboratories-US, Ltd. 16
Blood, culture Luminex..... 15	CA15-3 Randox Laboratories-US, Ltd. 16	Cannabinoid (marijuana), qualitative, urine Carolina Liquid Chemistries Corp. 13 Randox Laboratories-US, Ltd. 16	Chemistry analyzers Carolina Liquid Chemistries Corp. 13 Randox Laboratories-US, Ltd. 16
Blood, cytometry BD Biosciences..... 12 Techcyte 17	CA19-9 Randox Laboratories-US, Ltd. 16	Cannabinoids, assays Randox Laboratories-US, Ltd. 16	Chemistry controls Carolina Liquid Chemistries Corp. 13 Randox Laboratories-US, Ltd. 16
Blood, glucose (See Glucose) Carolina Liquid Chemistries Corp. 13	Calcium SEKISUI Diagnostics 16	Carbamazepine, serum Carolina Liquid Chemistries Corp. 13	Chemistry products and services Carolina Liquid Chemistries Corp. 13 College of American Pathologists 13 Denline Uniforms..... 14 Randox Laboratories-US, Ltd. 16
Blood, leukocyte differential Techcyte 17	Calcium assays (Electrolytes, analysis), standard Randox Laboratories-US, Ltd. 16	Carbon dioxide assays Randox Laboratories-US, Ltd. 16	Chemistry, diagnostics Carolina Liquid Chemistries Corp. 13 DiaSorin Molecular..... 14 Randox Laboratories-US, Ltd. 16
Blood, screening Techcyte 17	Calcium assays (See electrolytes, analysis) Randox Laboratories-US, Ltd. 16	Carcinoembryonic antigen (CEA) Randox Laboratories-US, Ltd. 16	Chemistry, special College of American Pathologists 13 Randox Laboratories-US, Ltd. 16
Blood, specialty controls Randox Laboratories-US, Ltd. 16	Calcium ISE Randox Laboratories-US, Ltd. 16	Cardiac markers Carolina Liquid Chemistries Corp. 13 College of American Pathologists 13 Randox Laboratories-US, Ltd. 16	Chimerism Analysis Software SoftGenetics 17
Blood, urea nitrogen (See BUN) Carolina Liquid Chemistries Corp. 13	Calcium, serum Carolina Liquid Chemistries Corp. 13	Cardiac markers, controls Randox Laboratories-US, Ltd. 16	Chlamydia trachomatis by DNA probe DiaSorin Molecular..... 14
Bordetella parapertussis detection DiaSorin Molecular..... 14 Luminex..... 15	Calcium, urine Carolina Liquid Chemistries Corp. 13	Cardiac risk assessment Randox Laboratories-US, Ltd. 16	Chloride, serum Carolina Liquid Chemistries Corp. 13
Bordetella pertussis and B. parapertussis by PCR DiaSorin Molecular..... 14 Luminex..... 15	Calibration kits College of American Pathologists 13 Luminex..... 15 Randox Laboratories-US, Ltd. 16 WSLH Proficiency Testing 17	Cardiac risk assessment (See Apolipoproteins; fibrinogen; Lp (a)) Randox Laboratories-US, Ltd. 16	Chloride, urine Carolina Liquid Chemistries Corp. 13
Bordetella pertussis and bordetella parapertussis DNA, real-time PCR Luminex..... 15	Calibration verifiers College of American Pathologists 13 WSLH Proficiency Testing 17	Cardiac testing College of American Pathologists 13 Randox Laboratories-US, Ltd. 16	Cholesterol, total Carolina Liquid Chemistries Corp. 13 SEKISUI Diagnostics 16
Bordetella pertussis detection DiaSorin Molecular..... 14 Luminex..... 15	Calibrators Randox Laboratories-US, Ltd. 16	Cardiovascular products and services Randox Laboratories-US, Ltd. 16	Cholesterol (CHOL) Carolina Liquid Chemistries Corp. 13 Randox Laboratories-US, Ltd. 16
Bordetella pertussis, nasopharyngeal culture, K Luminex..... 15	Calibrators, custom Randox Laboratories-US, Ltd. 16	CD34 cells, measurement BD Biosciences..... 12	Cholesterol testing Carolina Liquid Chemistries Corp. 13 Randox Laboratories-US, Ltd. 16
BUN Carolina Liquid Chemistries Corp. 13 Randox Laboratories-US, Ltd. 16	Campylobacter detection Luminex..... 15	CD4/CD8 cells BD Biosciences..... 12	
BUN (Urea) Carolina Liquid Chemistries Corp. 13	Cancer (See Oncology products and services) Randox Laboratories-US, Ltd. 16	CD4/CD8 cells, measurement BD Biosciences..... 12	
BUN, standard Randox Laboratories-US, Ltd. 16	Cancer (See Oncology products and services), antigens Randox Laboratories-US, Ltd. 16	CD4/CD8 ratio profile BD Biosciences..... 12	
Buprenorphine screen, urine Carolina Liquid Chemistries Corp. 13	Cancer markers BD Biosciences..... 12 College of American Pathologists 13 Randox Laboratories-US, Ltd. 16	Cell counting (See Flow cytometry) BD Biosciences..... 12	

Page	Page	Page	Page
Cholesterol, aqueous, standard	Globe Scientific..... 14	Complement 3 (C3)	Computer systems
Carolina Liquid	Hardy Diagnostics..... 14	Randox Laboratories-US, Ltd..... 16	Advanced Data Systems
Chemistries Corp..... 13	Clinical pathology	Complement 4 (C4)	Corporation..... 12
Cholesterol, liquid	College of American	Randox Laboratories-US, Ltd..... 16	CompuGroup Medical US..... 14
Carolina Liquid	Pathologists..... 13	Complement C2	Luminex..... 15
Chemistries Corp..... 13	Randox Laboratories-US, Ltd..... 16	Randox Laboratories-US, Ltd..... 16	Congenital disease
Cholesterol, total	Clinical toxicology	Complement C3, serum	DiaSorin Molecular..... 14
Carolina Liquid	College of American	Carolina Liquid	Conjugates
Chemistries Corp..... 13	Pathologists..... 13	Chemistries Corp..... 13	BD Biosciences..... 12
Randox Laboratories-US, Ltd..... 16	Randox Laboratories-US, Ltd..... 16	Complement C4, serum	Randox Laboratories-US, Ltd..... 16
Cholinesterase (CHE)	Clostridium difficile (Toxin B, cytotoxin)	Carolina Liquid	Conjugates, affinity purified
Carolina Liquid	Luminex..... 15	Chemistries Corp..... 13	BD Biosciences..... 12
Chemistries Corp..... 13	Clostridium difficile detection (See Toxin A detection)	Complete blood count (CBC)	Randox Laboratories-US, Ltd..... 16
Chronic leukemia profile, flow cytometry	Luminex..... 15	Techcyte..... 17	Conjugates, universal
BD Biosciences..... 12	Clostridium difficile toxin A assays by EIA	Complete blood count (CBC) with differential (with platelet count)	Randox Laboratories-US, Ltd..... 16
Chronic lymphocytic leukemia/lymphoma profile	Luminex..... 15	Techcyte..... 17	Consultative and expert services
BD Biosciences..... 12	Coagulation assays, products and services	Complete blood count (CBC) with differential (without platelet count)	Clinical Software Solutions..... 13
Chronic myelogenous leukemia profile	College of American	Techcyte..... 17	Consulting
BD Biosciences..... 12	Pathologists..... 13	Complete blood count (CBC) without differential (with platelet count)	Clinical Software Solutions..... 13
CK	Randox Laboratories-US, Ltd..... 16	Techcyte..... 17	Continuing education
Carolina Liquid	Cocaine	Complete blood count (CBC) without differential (without platelet count)	College of American
Chemistries Corp..... 13	Randox Laboratories-US, Ltd..... 16	Techcyte..... 17	Pathologists..... 13
CK (Creatine kinase)	Cocaine and benzoylecgonine, blood	Complete blood count (CBC) without differential (without platelet count)	WSLH Proficiency Testing..... 17
Carolina Liquid	Randox Laboratories-US, Ltd..... 16	Techcyte..... 17	Continuing medical education
Chemistries Corp..... 13	Cocaine metabolite confirmation, quantitative, urine	Complete blood count, HCT	College of American
Randox Laboratories-US, Ltd..... 16	Randox Laboratories-US, Ltd..... 16	Techcyte..... 17	Pathologists..... 13
SEKISUI Diagnostics..... 16	Cocaine metabolite screen and confirmation, blood	Complete blood count, HGB	Contract manufacturing
CK (IFCC)	Randox Laboratories-US, Ltd..... 16	Techcyte..... 17	Puritan Medical Products..... 16
Randox Laboratories-US, Ltd..... 16	Cocaine metabolite, qualitative, urine	Complete blood count, MCH	Randox Laboratories-US, Ltd..... 16
CK/MB (See Creatine kinase)	Carolina Liquid	Techcyte..... 17	Control, quality
Randox Laboratories-US, Ltd..... 16	Chemistries Corp..... 13	Complete blood count, MCHC	Bio-Rad Laboratories..... 13
Cleanrooms	Randox Laboratories-US, Ltd..... 16	Techcyte..... 17	Clinical Software Solutions..... 13
Denline Uniforms..... 14	Cocaine, assays	Complete blood count, MCV	Randox Laboratories-US, Ltd..... 16
Puritan Medical Products..... 16	Randox Laboratories-US, Ltd..... 16	Techcyte..... 17	Controls
Clinical chemistry, automated	Cocaine, metabolites, assays	Complete blood count, MPV	Randox Laboratories-US, Ltd..... 16
Randox Laboratories-US, Ltd..... 16	Randox Laboratories-US, Ltd..... 16	Techcyte..... 17	Controls, autoimmune
Clinical chemistry, automated, controls, reagents	Competency (See Educational products and services)	Complete blood count, PLT	Bio-Rad Laboratories..... 13
Randox Laboratories-US, Ltd..... 16	College of American	Techcyte..... 17	Controls, blood screening
Clinical chemistry, controls	Pathologists..... 13	Complete blood count, RDW	Bio-Rad Laboratories..... 13
Bio-Rad Laboratories..... 13	StaffReady..... 17	Techcyte..... 17	Controls, cardiac marker
Carolina Liquid	WSLH Proficiency Testing..... 17	Computer software	Bio-Rad Laboratories..... 13
Chemistries Corp..... 13	Competency assessment program	Advanced Data Systems	Randox Laboratories-US, Ltd..... 16
Randox Laboratories-US, Ltd..... 16	College of American	Corporation..... 12	Controls, coagulation assays
SEKISUI Diagnostics..... 16	Pathologists..... 13	Clinical Software Solutions..... 13	Bio-Rad Laboratories..... 13
Clinical chemistry, reagents	StaffReady..... 17	CompuGroup Medical US..... 14	Randox Laboratories-US, Ltd..... 16
Carolina Liquid	WSLH Proficiency Testing..... 17	Luminex..... 15	Controls, custom
Chemistries Corp..... 13	Complement 3 (C3)	SoftGenetics..... 17	Bio-Rad Laboratories..... 13
Randox Laboratories-US, Ltd..... 16	Randox Laboratories-US, Ltd..... 16	StaffReady..... 17	Randox Laboratories-US, Ltd..... 16
SEKISUI Diagnostics..... 16	Complement 4 (C4)	Computer systems	Controls, diabetes
Clinical microscopy	Randox Laboratories-US, Ltd..... 16	Advanced Data Systems	Bio-Rad Laboratories..... 13
College of American	Complement C2	Corporation..... 12	Randox Laboratories-US, Ltd..... 16
Pathologists..... 13	Randox Laboratories-US, Ltd..... 16	Clinical Software Solutions..... 13	

Peer-Reviewed Insight for Clinical Laboratory Professionals

At Your Fingertips



**Complimentary
Subscription @**
MLO-ONLINE.COM/SUBSCRIBE



Print



eNews



Digital Edition

Page	Page	Page	Page
Controls, drugs of abuse	Controls, tumor marker	C-Reactive protein, high sensitivity (CRPHS)	Cyclospora smear, stool
Bio-Rad Laboratories 13	Bio-Rad Laboratories 13	Randox Laboratories-US, Ltd. 16	Techcyte 17
Randox Laboratories-US, Ltd. 16	Randox Laboratories-US, Ltd. 16	C-reactive protein, nephelometry kit	Cyptosporidium and Isospora Smear, stool
Controls, general chemistry	Controls, urinalysis	Randox Laboratories-US, Ltd. 16	Techcyte 17
Bio-Rad Laboratories 13	Bio-Rad Laboratories 13	Creatine kinase (CK) and lactic acid dehydrogenase (LD) isoenzymes	Cystatin C
Randox Laboratories-US, Ltd. 16	Randox Laboratories-US, Ltd. 16	Randox Laboratories-US, Ltd. 16	Carolina Liquid
Controls, hematology	Controls, urine	Creatine kinase (CK), MB	Chemistries Corp. 13
Bio-Rad Laboratories 13	Bio-Rad Laboratories 13	Randox Laboratories-US, Ltd. 16	Randox Laboratories-US, Ltd. 16
Randox Laboratories-US, Ltd. 16	Randox Laboratories-US, Ltd. 16	Creatine kinase (CK), MB and total	Cystic Fibrosis (ACOG Screen)
Controls, hemoglobinopathy	Controls, urine chemistry	Randox Laboratories-US, Ltd. 16	Luminex 15
Bio-Rad Laboratories 13	Bio-Rad Laboratories 13	Creatine kinase (CK), total, serum	Cystic Fibrosis Analysis Software
Randox Laboratories-US, Ltd. 16	Randox Laboratories-US, Ltd. 16	Carolina Liquid	SoftGenetics 17
Controls, hormone	Controls, virology	Chemistries Corp. 13	Cystic Fibrosis Profile
Bio-Rad Laboratories 13	Bio-Rad Laboratories 13	Randox Laboratories-US, Ltd. 16	Luminex 15
Controls, immunoassays	COVID-19 - Nucleic acid amplification tests (NAAT)	Creatine kinase assays (CK)	Cysticercosis (taenia solium)
Bio-Rad Laboratories 13	Luminex 15	Carolina Liquid	Techcyte 17
Randox Laboratories-US, Ltd. 16	COVID-19 - Multiplex PCR Test	Chemistries Corp. 13	Cytochrome P450 2C19 Genotyping
Controls, immunologic	DiaSorin Molecular 14	Randox Laboratories-US, Ltd. 16	Luminex 15
Bio-Rad Laboratories 13	Luminex 15	SEKISUI Diagnostics 16	Cytogenetics
Controls, infectious disease	COVID-19 - Nucleic Acid Multiplex Test	Creatine kinase assays, standard	College of American
Bio-Rad Laboratories 13	Luminex 15	Randox Laboratories-US, Ltd. 16	Pathologists 13
Denline Uniforms 14	COVID-19 - POC Immunoassay	Creatine, serum	Cytokines, assays
Randox Laboratories-US, Ltd. 16	Carolina Liquid	Randox Laboratories-US, Ltd. 16	Randox Laboratories-US, Ltd. 16
Controls, ligand	Chemistries Corp. 13	Creatinine (CRE)	Cytology products and services
Bio-Rad Laboratories 13	COVID-19 - Qualitative Assay	Randox Laboratories-US, Ltd. 16	College of American
Controls, lipid	Carolina Liquid	Creatinine, assays	Pathologists 13
Bio-Rad Laboratories 13	Chemistries Corp. 13	Randox Laboratories-US, Ltd. 16	Puritan Medical Products 16
Randox Laboratories-US, Ltd. 16	Luminex 15	Creatinine, serum	Cytomegalovirus (CMV) DNA by PCR, quantitative
Controls, molecular	COVID-19 - Rapid Diagnostic Test	Carolina Liquid	DiaSorin Molecular 14
Bio-Rad Laboratories 13	Carolina Liquid	Chemistries Corp. 13	Cytomegalovirus (CMV) DNA detection and quantitation
Controls, nucleic acid	Chemistries Corp. 13	Hardy Diagnostics 14	DiaSorin Molecular 14
Bio-Rad Laboratories 13	COVID-19 - Real-time RT-PCR Test	Creatinine, standard	Cytopathology
Controls, peptides	DiaSorin Molecular 14	Randox Laboratories-US, Ltd. 16	College of American
Randox Laboratories-US, Ltd. 16	Luminex 15	Criminalistic services	Pathologists 13
Controls, protein	COVID-19 - Serological POC Test	SoftGenetics 17	Cytoplasmic antibodies
Bio-Rad Laboratories 13	Carolina Liquid	Cryptosporidium detection	BD Biosciences 12
Randox Laboratories-US, Ltd. 16	Chemistries Corp. 13	Techcyte 17	
Controls, respiratory	C-Reactive protein (CRP)	Cryptosporidium smear, stool	D
Bio-Rad Laboratories 13	Carolina Liquid	Techcyte 17	
Controls, serum chemistry	Chemistries Corp. 13	CSF test kits	Data management
Bio-Rad Laboratories 13	Randox Laboratories-US, Ltd. 16	Randox Laboratories-US, Ltd. 16	Advanced Data Systems
Randox Laboratories-US, Ltd. 16	SEKISUI Diagnostics 16	Culture identification	Corporation 12
Controls, speciality	C-Reactive protein (CRP), quantitative	College of American	Bio-Rad Laboratories 13
Bio-Rad Laboratories 13	Randox Laboratories-US, Ltd. 16	Pathologists 13	Clinical Software Solutions 13
Randox Laboratories-US, Ltd. 16	C-Reactive protein, cardiac risk (hsCRP)	Hardy Diagnostics 14	College of American
Controls, thyronines	Carolina Liquid	Custom product development and manufacturing	Pathologists 13
Bio-Rad Laboratories 13	Chemistries Corp. 13	Clinical Software Solutions 13	CompuGroup Medical US 14
Randox Laboratories-US, Ltd. 16	Randox Laboratories-US, Ltd. 16	Randox Laboratories-US, Ltd. 16	Orchard Software Corporation 16
Controls, toxicology			D-dimer
Bio-Rad Laboratories 13			Carolina Liquid
Randox Laboratories-US, Ltd. 16			Chemistries Corp. 13

Page	Page	Page	Page
D-dimer (See Fibrin, degradation products, assays)	Direct enzymatic HbA1c assays	Drug abuse profile (routine), urine (eight drugs) (Includes GC/MS confirmation)	Ecstasy assays
Randox Laboratories-US, Ltd. 16	Carolina Liquid Chemistries Corp. 13	Randox Laboratories-US, Ltd. 16	Carolina Liquid Chemistries Corp. 13
D-dimer, automated	Direct HDL cholesterol	Drug abuse profile, urine (eight drugs and alcohol)	Randox Laboratories-US, Ltd. 16
Randox Laboratories-US, Ltd. 16	Randox Laboratories-US, Ltd. 16	Randox Laboratories-US, Ltd. 16	Educational products and services
Decapping sample collection vials	Direct LDL cholesterol	Drug profile, blood (five drugs)	College of American Pathologists 13
LGP (Laboratory Growth & Productivity) 15	Randox Laboratories-US, Ltd. 16	Randox Laboratories-US, Ltd. 16	WSLH Proficiency Testing 17
Dengue fever, DNA	Disposable lab coats	Drug profile, blood (seven drugs)	EGFR
DiaSorin Molecular 14	Denline Uniforms 14	Randox Laboratories-US, Ltd. 16	Randox Laboratories-US, Ltd. 16
Dengue fever, screening tests	Hardy Diagnostics 14	Drug profile, qualitative, urine or gastric contents (with cannabinoids)	EIA (See Enzyme immunoassays)
DiaSorin Molecular 14	Disposable laboratory products	Randox Laboratories-US, Ltd. 16	DiaSorin Molecular 14
Dermatopathology	Denline Uniforms 14	Drug profile, urine (ten drugs) by immunoassays with GC/MS confirmation	Electrolyte panel
College of American Pathologists 13	Hardy Diagnostics 14	Randox Laboratories-US, Ltd. 16	Randox Laboratories-US, Ltd. 16
Device connectivity	DNA	Drug profile, urine (comprehensive (with volatiles))	Electrophoresis
Clinical Software Solutions 13	Luminex 15	Randox Laboratories-US, Ltd. 16	Randox Laboratories-US, Ltd. 16
CompuGroup Medical US 14	DNA Content/Cell Cycle Analysis	Drug profile, urine (comprehensive (without volatiles))	Electrophoresis, systems
Orchard Software Corporation 16	BD Biosciences 12	Randox Laboratories-US, Ltd. 16	SoftGenetics 17
Diabetes mellitus	DNA ploidy analysis	Drug susceptibility testing	ELISA systems
KRONUS, Inc. 15	BD Biosciences 12	College of American Pathologists 13	Randox Laboratories-US, Ltd. 16
Randox Laboratories-US, Ltd. 16	College of American Pathologists 13	Drugs of abuse detection	ELISA systems, reagents
Diabetes mellitus, management	DNA ploidy analysis, body fluids rare	College of American Pathologists 13	DiaSorin Molecular 14
Randox Laboratories-US, Ltd. 16	BD Biosciences 12	Drugs of abuse detection, controls	Randox Laboratories-US, Ltd. 16
Diabetes mellitus, management, controls	DNA ploidy analysis, frozen tissue	Randox Laboratories-US, Ltd. 16	ELISA systems, reference control serum
Randox Laboratories-US, Ltd. 16	BD Biosciences 12	Drugs of abuse detection, immunoassays	Randox Laboratories-US, Ltd. 16
Diagnostic services and products	DNA ploidy analysis, needle biopsy tissue sections rare	Randox Laboratories-US, Ltd. 16	ELISA systems, urine
Puritan Medical Products 16	BD Biosciences 12	Drugs of abuse detection, on-site kits	Randox Laboratories-US, Ltd. 16
Randox Laboratories-US, Ltd. 16	DNA ploidy analysis, paraffin block	Globe Scientific 14	Endocrinology products and services
Diagnostic services, testing, salivary	BD Biosciences 12	Randox Laboratories-US, Ltd. 16	College of American Pathologists 13
Randox Laboratories-US, Ltd. 16	DNA, amplification products	Drugs of abuse detection, whole blood	Endocrinology products and services (See Thyroid hormone assays)
Differential WBC count	DiaSorin Molecular 14	Randox Laboratories-US, Ltd. 16	KRONUS, Inc. 15
Techcyte 17	DNA, reagent kit	Drugs of abuse detection, whole blood	Randox Laboratories-US, Ltd. 16
Digitoxin, assays and therapeutic drug monitoring	DiaSorin Molecular 14	Drugs of abuse detection, whole blood	Entamoeba detection
Randox Laboratories-US, Ltd. 16	DNA/RNA based, reagents	Randox Laboratories-US, Ltd. 16	Techcyte 17
Digitoxin, serum	DiaSorin Molecular 14	Drugs of abuse detection, whole blood	Enterobacteriaceae identification tests
Randox Laboratories-US, Ltd. 16	Randox Laboratories-US, Ltd. 16	Drugs of abuse detection, whole blood	Hardy Diagnostics 14
Digoxin, assays and therapeutic drug monitoring (DIG)	DNA/RNA based, systems	Randox Laboratories-US, Ltd. 16	Enterohemorrhagic E. coli (EHEC) culture
Randox Laboratories-US, Ltd. 16	Puritan Medical Products 16	Drugs of abuse detection, whole blood	Hardy Diagnostics 14
Digoxin, serum	Drug abuse profile (HHS/DOT), urine (five drugs)	Randox Laboratories-US, Ltd. 16	Enterohemorrhagic E. coli (EHEC) shiga toxin, EIA
Carolina Liquid Chemistries Corp. 13	Randox Laboratories-US, Ltd. 16	Drugs of abuse detection, whole blood	Hardy Diagnostics 14
Randox Laboratories-US, Ltd. 16	Drug abuse profile (HHS/DOT), urine (five drugs) (Includes GC/MS confirmation)	Randox Laboratories-US, Ltd. 16	Enterovirus detection by RT-PCR
Direct bilirubin (DBIL)	Randox Laboratories-US, Ltd. 16	Drugs of abuse detection, whole blood	DiaSorin Molecular 14
Carolina Liquid Chemistries Corp. 13	Drug abuse profile (routine), urine (eight drugs)	Randox Laboratories-US, Ltd. 16	Luminex 15
	Randox Laboratories-US, Ltd. 16		Enterovirus testing
			DiaSorin Molecular 14
			Luminex 15

Page

Enterovirus, real-time PCR

DiaSorin Molecular..... 14

Luminex..... 15

Enzyme immunoassays

Randox Laboratories-US, Ltd..... 16

Enzyme immunoassays, automated

Randox Laboratories-US, Ltd..... 16

Enzyme immunoassays, controls

Randox Laboratories-US, Ltd..... 16

Enzyme immunoassays, instrumentation

Randox Laboratories-US, Ltd..... 16

Enzymes, research use

Randox Laboratories-US, Ltd..... 16

Epidermal growth factor receptor (EGFR) pharmD (HER-1) oncoprotein, paraffin-embedded colorectal tissue

Randox Laboratories-US, Ltd..... 16

Epidermal growth factor receptor (EGFR), paraffin block

Randox Laboratories-US, Ltd..... 16

Epstein-Barr virus (EBV), DNA probe

DiaSorin Molecular..... 14

Error prevention

Clinical Software Solutions..... 13

College of American Pathologists..... 13

Escherichia coli

Hardy Diagnostics..... 14

Luminex..... 15

Escherichia coli, 0157:H7

Luminex..... 15

Escherichia coli, shiga toxin

Luminex..... 15

Esoteric testing

College of American Pathologists..... 13

Estradiol

Randox Laboratories-US, Ltd..... 16

Estradiol assays

Randox Laboratories-US, Ltd..... 16

Ethanol assays (See Alcohol assays)

SEKISUI Diagnostics..... 16

F

Factor V Leiden mutation analysis

DiaSorin Molecular..... 14

Page

Factor V, Leiden mutation DNA test

DiaSorin Molecular..... 14

Fatty acids, free (nonester)

Randox Laboratories-US, Ltd..... 16

Fecal occult blood (FOB) (See Stool, occult blood)

Hardy Diagnostics..... 14

Fecal occult blood (iFOB)

Carolina Liquid Chemistries Corp..... 13

SEKISUI Diagnostics..... 16

Fentanyl

Carolina Liquid Chemistries Corp..... 13

Ferritin assay (FERR)

Randox Laboratories-US, Ltd..... 16

Fibrinogen

Randox Laboratories-US, Ltd..... 16

Fibrinogen, assays

Randox Laboratories-US, Ltd..... 16

Fine-needle aspiration cytology

College of American Pathologists..... 13

FITC

BD Biosciences..... 12

FITC-conjugates

BD Biosciences..... 12

Flow cytometry

College of American Pathologists..... 13

Folate (Folic acid), serum

Randox Laboratories-US, Ltd..... 16

Folate assays

Randox Laboratories-US, Ltd..... 16

Follicle-stimulating hormone (FSH) and luteinizing hormone (LH)

Randox Laboratories-US, Ltd..... 16

Follicle-stimulating hormone assays

Randox Laboratories-US, Ltd..... 16

Forensic toxicology

College of American Pathologists..... 13

Randox Laboratories-US, Ltd..... 16

Fragile X Analysis Software

SoftGenetics..... 17

Free fatty acids

Randox Laboratories-US, Ltd..... 16

Free Kappa and lambda light chains

Carolina Liquid Chemistries Corp..... 13

Page

Free T3 assays

Randox Laboratories-US, Ltd..... 16

Free T4 assays

Randox Laboratories-US, Ltd..... 16

Free T4/TSH

Randox Laboratories-US, Ltd..... 16

Fructosamine

Carolina Liquid Chemistries Corp..... 13

FT3

Randox Laboratories-US, Ltd..... 16

FT4

Randox Laboratories-US, Ltd..... 16

Fungal serology

College of American Pathologists..... 13

Fungus (mycology) culture

Hardy Diagnostics..... 14

Fungus culture with stain

Hardy Diagnostics..... 14

Fungus stain

Hardy Diagnostics..... 14

Furniture

Proline Laboratory Furniture..... 16

G

GAD65 (See Glutamic acid decarboxylase antibody kits)

KRONUS, Inc..... 15

Gamma-glutamyl transpeptidase (GGT)

SEKISUI Diagnostics..... 16

Genetic testing

College of American Pathologists..... 13

Genotyping Software

SoftGenetics..... 17

Gentamicin assay (GENT)

Carolina Liquid Chemistries Corp..... 13

Randox Laboratories-US, Ltd..... 16

GGT

Carolina Liquid Chemistries Corp..... 13

GGT assay

Carolina Liquid Chemistries Corp..... 13

Giardia lamblia detection

Luminex..... 15

Techcyte..... 17

Glassware

Globe Scientific..... 14

Glucose (GLU)

Carolina Liquid Chemistries Corp..... 13

Page

Randox Laboratories-US, Ltd..... 16

SEKISUI Diagnostics..... 16

Glucose (GLU), serum

Carolina Liquid Chemistries Corp..... 13

Glucose monitoring

Randox Laboratories-US, Ltd..... 16

Glucose, assays

Carolina Liquid Chemistries Corp..... 13

Randox Laboratories-US, Ltd..... 16

Glucose, body fluid

Carolina Liquid Chemistries Corp..... 13

Glucose, liquid

Carolina Liquid Chemistries Corp..... 13

Glucose, plasma

Carolina Liquid Chemistries Corp..... 13

Glucose, standard

Carolina Liquid Chemistries Corp..... 13

Randox Laboratories-US, Ltd..... 16

Glutamic acid decarboxylase (GAD) autoantibody

KRONUS, Inc..... 15

Glutamic acid decarboxylase antibody kits

KRONUS, Inc..... 15

Gram slide stain

Hardy Diagnostics..... 14

Gram's stain

Hardy Diagnostics..... 14

Techcyte..... 17

Granulocyte

BD Biosciences..... 12

Granulocyte antibodies

BD Biosciences..... 12

Granulocyte, analysis

BD Biosciences..... 12

Group A streptococcus

Hardy Diagnostics..... 14

Luminex..... 15

Group A streptococcus direct test, DNA probe

Luminex..... 15

Group B strep PCR

Luminex..... 15

Group B streptococcus colonization detection culture

Hardy Diagnostics..... 14

Growth factor assays

Randox Laboratories-US, Ltd..... 16

Page	Page	Page	Page
Gynecologic pap smear (one slide) Techcyte 17	Gynecologic Pap Test (Image-guided and non-image guided), Liquid-based Preparation and Chlamydia trachomatis by NAA With Reflex to HPV, High-risk DNA Detection When ASC-U, ASC-H, LSIL, HSIL, AGUS Techcyte 17	Gynecologic Pap Test (Image-guided and non-image guided), Liquid-based Preparation With Reflex to HPV, High-risk DNA Detection When ASC-U, ASC-H, LSIL, HSIL, AGUS Techcyte 17	Gynecologic Pap Test (Image-guided or non-image guided), Liquid-based Preparation and /Chlamydia/ Gonococcus/Trichomonas/ NAA With Reflex to Human Papillomavirus (HPV), High-risk DNA Detection When ASC-U With Reflex to Genotypes 16 and 18 Techcyte 17
Gynecologic pap smear (two slides) Techcyte 17	Gynecologic Pap Test (Image-guided and non-image guided), Liquid-based Preparation and Chlamydia/ Gonococcus by NAA Techcyte 17	Gynecologic Pap Test (Image-guided or non-image guided), Liquid-based Preparation and /Chlamydia/ Gonococcus/Trichomonas/ NAA With Reflex to Human Papillomavirus (HPV), High-risk DNA Detection When ASC-U, ASC-H, LSIL, HSIL, AGUS Techcyte 17	Gynecologic Pap Test (Image-guided or non-image guided), Liquid-based Preparation and /Chlamydia/ Gonococcus/Trichomonas/ NAA With Reflex to Human Papillomavirus (HPV), High-risk DNA Detection When ASC-U Techcyte 17
Gynecologic pap smear, liquid-based preparation & human papillomavirus (HPV) profile, Hybrid Capture II, high-risk Techcyte 17	Gynecologic Pap Test (Image-guided and non-image guided), Liquid-based Preparation and Chlamydia/ Gonococcus by NAA and HPV, High-risk DNA Detection Techcyte 17	Gynecologic Pap Test (Image-guided or non-image guided), Liquid-based Preparation and Chlamydia/ Gonococcus/Trichomonas NAA Techcyte 17	Gynecologic Pap Test (Image-guided or non-image guided), Liquid-based Preparation With Reflex to Human Papillomavirus (HPV), High-risk DNA Detection When ASC-U With Reflex to Genotypes 16 and 18 Techcyte 17
Gynecologic pap smear, liquid-based preparation (image-guided) and human papillomavirus (HPV) profile, Hybrid Capture II, high risk Techcyte 17	Gynecologic Pap Test (Image-guided and non-image guided), Liquid-based Preparation and Chlamydia/ Gonococcus by NAA With Reflex to HPV, High-risk DNA Detection When ASC-U Techcyte 17	Gynecologic Pap Test (Image-guided or non-image guided), Liquid-based Preparation and Chlamydia/ Gonococcus/Trichomonas NAA and Human Papillomavirus (HPV), High-risk DNA Detection Techcyte 17	Gynecologic pap test, liquid-based preparation (image-guided) Techcyte 17
Gynecologic pap smear, liquid-based preparation with reflex to high-risk human papillomavirus (HPV), Hybrid Capture II Techcyte 17	Gynecologic Pap Test (Image-guided and non-image guided), Liquid-based Preparation and Chlamydia/ Gonococcus by NAA With Reflex to HPV, High-risk DNA Detection When ASC-U Techcyte 17	Gynecologic Pap Test (Image-guided or non-image guided), Liquid-based Preparation and Chlamydia/ Gonococcus/Trichomonas/ NAA and Human Papillomavirus (HPV) High-risk DNA Detection With Reflex to Genotypes 16 and 18 Techcyte 17	Gynecologic pap test, liquid-based preparation and chlamydia trachomatis by nucleic acid amplification Techcyte 17
Gynecologic pap smear, liquid-based preparation with reflex to human papillomavirus (HPV), Hybrid Capture II Techcyte 17	Gynecologic Pap Test (Image-guided and non-image guided), Liquid-based Preparation and Chlamydia/ Gonococcus by NAA With Reflex to HPV, High-risk DNA Detection When ASC-U, ASC-H, LSIL, HSIL, AGUS Techcyte 17	Gynecologic Pap Test (Image-guided or non-image guided), Liquid-based Preparation and Chlamydia/ Gonococcus/Trichomonas/ NAA and Human Papillomavirus (HPV) High-risk DNA Detection With Reflex to HPV Genotypes 16 and 18 Techcyte 17	Gynecologic pap test, liquid-based preparation and chlamydia trachomatis/ neisseria gonorrhoeae profile with reflex to high-risk human papillomavirus (HPV) Techcyte 17
Gynecologic pap smear, mono-layer Techcyte 17	Gynecologic Pap Test (Image-guided and non-image guided), Liquid-based Preparation and Chlamydia/ Gonococcus by NAA With Reflex to HPV, High-risk DNA Detection When ASC-U, ASC-H, LSIL, HSIL, AGUS Techcyte 17		
Gynecologic Pap Test (Image-guided and non-image guided), Liquid-based Preparation Techcyte 17	Gynecologic Pap Test (Image-guided and non-image guided), Liquid-based Preparation and HPV, High-and Low-risk DNA Detection Techcyte 17		
Gynecologic Pap Test (Image-guided and non-image guided), Liquid-based Preparation and Chlamydia trachomatis by NAA Techcyte 17	Gynecologic Pap Test (Image-guided and non-image guided), Liquid-based Preparation With Maturation Index Techcyte 17		
Gynecologic Pap Test (Image-guided and non-image guided), Liquid-based Preparation and Chlamydia trachomatis by NAA With Reflex to HPV, High-risk DNA Detection When ASC-U Techcyte 17	Gynecologic Pap Test (Image-guided and non-image guided), Liquid-based Preparation With Reflex to HPV, High-risk DNA Detection When ASC-U Techcyte 17		

	Page
Gynecologic pap test, liquid-based preparation and chlamydia/gonococcus by nucleic acid amplification	
Techcyte	17
Gynecologic pap testing	
College of American Pathologists	13
Techcyte	17

H

H. pylori, serum or plasma	
SEKISUI Diagnostics	16
H. pylori, whole blood	
SEKISUI Diagnostics	16
Hantavirus testing	
DiaSorin Molecular	14
Haptoglobin assays (HAPT)	
Carolina Liquid Chemistries Corp.	13
hCG (Human chorionic gonadotropin)	
SEKISUI Diagnostics	16
hCG Ultra Combo	
SEKISUI Diagnostics	16
HDL	
Carolina Liquid Chemistries Corp.	13
Helper T-lymphocyte marker CD4	
BD Biosciences	12
Helper/suppressor and natural killer profile	
BD Biosciences	12
Hematology products and services	
College of American Pathologists	13
Denline Uniforms	14
Hardy Diagnostics	14
Hematology products and services, analyzers	
Carolina Liquid Chemistries Corp.	13
Hematology special staining program	
Techcyte	17
Hematology test reports	
Techcyte	17
Hematopathology	
BD Biosciences	12
College of American Pathologists	13
Hematopathology profile 1, acute leukemia or MDS assessment	
BD Biosciences	12

	Page
Hematopathology profile 2, acute promyelocytic leukemia assessment	
BD Biosciences	12
Hematopathology profile 3, acute promyelocytic leukemia follow-up	
BD Biosciences	12
Hematopathology profile 4, CML assessment	
BD Biosciences	12
Hemoglobin (Hgb) A1c	
Carolina Liquid Chemistries Corp.	13
Randox Laboratories-US, Ltd.	16
Hemoglobin A1c	
Randox Laboratories-US, Ltd.	16
Hemoglobin HbA_c	
Randox Laboratories-US, Ltd.	16
Hemoglobin, glycated assay	
Randox Laboratories-US, Ltd.	16
Herpes simplex virus (HSV) by DNA probe	
DiaSorin Molecular	14
Herpes simplex virus (HSV) by PCR probe and quant PCR	
DiaSorin Molecular	14
Herpes simplex virus (HSV) types I/II, DNA, PCR	
DiaSorin Molecular	14
Luminex	15
Herpes simplex virus testing	
Luminex	15
Herpes virus testing	
Luminex	15
HerpeSelect, type-specific HSV antibody	
DiaSorin Molecular	14
HHV6	
DiaSorin Molecular	14
High density lipoprotein (HDL) cholesterol	
Randox Laboratories-US, Ltd.	16
High sensitivity C-reactive protein (hs-CRP)	
Randox Laboratories-US, Ltd.	16
High-density lipoprotein	
Randox Laboratories-US, Ltd.	16
SEKISUI Diagnostics	16
High-density lipoprotein, control	
Randox Laboratories-US, Ltd.	16
High-density lipoprotein, direct	
Randox Laboratories-US, Ltd.	16

	Page
High-density lipoprotein, serum	
Randox Laboratories-US, Ltd.	16
High-density lipoprotein, whole blood	
Randox Laboratories-US, Ltd.	16
HIPAA compliance	
Clinical Software Solutions	13
Histocompatibility testing	
College of American Pathologists	13
Histology	
College of American Pathologists	13
Histology Special Stains, Microorganisms	
Hardy Diagnostics	14
Histopathology	
College of American Pathologists	13
HLA	
BD Biosciences	12
HLA A-specific antigen	
BD Biosciences	12
HLA B-specific antigen	
BD Biosciences	12
HLA DR transplant	
BD Biosciences	12
HLA DR-specific antigen	
BD Biosciences	12
HLA typing (A, B Loci)	
BD Biosciences	12
HLA typing (A, B, and C Loci)	
BD Biosciences	12
HLA-B27	
BD Biosciences	12
HLA-B27 analysis	
BD Biosciences	12
Homocyst(e)ine, plasma or serum	
Carolina Liquid Chemistries Corp.	13
Randox Laboratories-US, Ltd.	16
Homocysteine assay	
Carolina Liquid Chemistries Corp.	13
Randox Laboratories-US, Ltd.	16
hsCRP	
Carolina Liquid Chemistries Corp.	13
Human anti-mouse antibody	
BD Biosciences	12
Human Identity Software	
SoftGenetics	17

	Page
I	
IA-2 antibody kits	
KRONUS, Inc.	15
IA-2 autoantibodies	
KRONUS, Inc.	15
IFA (See Immunofluorescence assays)	
BD Biosciences	12
IgA	
BD Biosciences	12
Carolina Liquid Chemistries Corp.	13
Randox Laboratories-US, Ltd.	16
IgE	
BD Biosciences	12
Randox Laboratories-US, Ltd.	16
IgG	
BD Biosciences	12
Carolina Liquid Chemistries Corp.	13
Hardy Diagnostics	14
Randox Laboratories-US, Ltd.	16
IgM	
BD Biosciences	12
Carolina Liquid Chemistries Corp.	13
Immunoassays	
BD Biosciences	12
Randox Laboratories-US, Ltd.	16
Immunoassays, analyzers	
Carolina Liquid Chemistries Corp.	13
Randox Laboratories-US, Ltd.	16
Immunoassays, automated systems	
Randox Laboratories-US, Ltd.	16
Immunoassays, controls	
Randox Laboratories-US, Ltd.	16
Immunoassays, instrumentation	
Randox Laboratories-US, Ltd.	16
Immunoassays, rapid	
SEKISUI Diagnostics	16
Immunoassays, reagents	
Randox Laboratories-US, Ltd.	16
Immunoblot kits	
DiaSorin Molecular	14
Immunochemistry	
BD Biosciences	12
DiaSorin Molecular	14
Randox Laboratories-US, Ltd.	16
Immunocyt monitoring for bladder cancer	
Techcyte	17
ImmunoCyt/urine cytology profile	
Techcyte	17

	Page		Page		Page		Page
Immunocytochemistry		Influenza Virus Types A and B, RSV, Real-time PCR		Iron (IRON), serum		Laboratory automation (See also Automated analyzers)	
BD Biosciences.....	12	Luminex.....	15	Carolina Liquid		Advanced Data Systems	
Immunodiagnosics		Information management		Chemistries Corp.....	13	Corporation.....	12
Randox Laboratories-US, Ltd.....	16	Advanced Data Systems		Iron and total Iron-binding capacity (TIBC)		LGP (Laboratory Growth & Productivity).....	15
Immunofluorescence assays		Corporation.....	12	Carolina Liquid		Laboratory automation/robotics	
BD Biosciences.....	12	Clinical Software Solutions.....	13	Chemistries Corp.....	13	LGP (Laboratory Growth & Productivity).....	15
Immunohistochemistry		CompuGroup Medical US.....	14	Iron assays (IRON)		Laboratory information systems	
BD Biosciences.....	12	StaffReady.....	17	Carolina Liquid		Clinical Software Solutions.....	13
College of American Pathologists.....	13	Information systems		Chemistries Corp.....	13	CompuGroup Medical US.....	14
Immunohistochemistry Antibody menu		Advanced Data Systems		Randox Laboratories-US, Ltd.....	16	Orchard Software Corporation....	16
BD Biosciences.....	12	Corporation.....	12	Iron-binding capacity		Laboratory process analysis	
Immunology products and services		Clinical Software Solutions.....	13	Carolina Liquid		Clinical Software Solutions.....	13
College of American Pathologists.....	13	CompuGroup Medical US.....	14	Chemistries Corp.....	13	College of American Pathologists.....	13
Denline Uniforms.....	14	Insulin		Randox Laboratories-US, Ltd.....	16	Laboratory safety	
Randox Laboratories-US, Ltd.....	16	Carolina Liquid		Islet cell antibody		College of American Pathologists.....	13
Immunology products and services, controls		Chemistries Corp.....	13	KRONUS, Inc.....	15	Denline Uniforms.....	14
Randox Laboratories-US, Ltd.....	16	Insulin autoantibodies (IAA)		ISO 15189 accreditation		LGP (Laboratory Growth & Productivity).....	15
Individualized Quality Control Plan (IQCP) Implementation Tools		KRONUS, Inc.....	15	College of American Pathologists.....	13	Laboratory supplies	
Clinical Software Solutions.....	13	Insulin, autoantibody kits		ISO 15189 Medical Laboratory Accreditation Services		Denline Uniforms.....	14
College of American Pathologists.....	13	KRONUS, Inc.....	15	College of American Pathologists.....	13	Globe Scientific.....	14
Infection control		Insulin-like growth factor I (IGF-I)		ISO 15189 Medical Laboratory Accreditation Services		Hardy Diagnostics.....	14
Denline Uniforms.....	14	Randox Laboratories-US, Ltd.....	16	College of American Pathologists.....	13	Randox Laboratories-US, Ltd.....	16
Hardy Diagnostics.....	14	Insulin-like growth factor II (IGF-II)		K		Lactate dehydrogenase assays (IFCC) (LDH)	
Infectious disease		Randox Laboratories-US, Ltd.....	16	Kappa chain		Carolina Liquid	
College of American Pathologists.....	13	Insulin-like Growth Factor-binding Protein 2 (IGFBP-2)		BD Biosciences.....	12	Chemistries Corp.....	13
Randox Laboratories-US, Ltd.....	16	Randox Laboratories-US, Ltd.....	16	Ketone bodies, serum		SEKISUI Diagnostics.....	16
Infectious disease, testing		Interferon beta, antibody test		Randox Laboratories-US, Ltd.....	16	Lambda chain	
DiaSorin Molecular.....	14	Randox Laboratories-US, Ltd.....	16	Kidney function, assessment		BD Biosciences.....	12
Influenza A and B antibodies, quantitative		Interleukin beta, antibody test		Randox Laboratories-US, Ltd.....	16	Latex agglutination tests	
SEKISUI Diagnostics.....	16	Randox Laboratories-US, Ltd.....	16	Kits, enzyme immunoassays		Hardy Diagnostics.....	14
Influenza A and B by direct immunoassay		Interleukin-12 (IL-12)		Randox Laboratories-US, Ltd.....	16	LDH	
SEKISUI Diagnostics.....	16	Randox Laboratories-US, Ltd.....	16	L		Carolina Liquid	
Influenza A and B, Real-time RT-PCR With Reflex to Influenza A, H1N1 (Novel)		Interleukin-2		Lab Benches		Chemistries Corp.....	13
Luminex.....	15	Randox Laboratories-US, Ltd.....	16	Proline Laboratory Furniture.....	16	LDH (Lactate dehydrogenase)	
Influenza Flu Control Kit		Interleukin-2 (IL-2)		Lab Coats		Carolina Liquid	
SEKISUI Diagnostics.....	16	Randox Laboratories-US, Ltd.....	16	Denline Uniforms.....	14	Chemistries Corp.....	13
Influenza Flu Test Kit		Interleukin-2 receptor (IL-2R)		Hardy Diagnostics.....	14	LDL	
Luminex.....	15	Randox Laboratories-US, Ltd.....	16	Lab portals		Carolina Liquid	
SEKISUI Diagnostics.....	16	Interleukin-6		Clinical Software Solutions.....	13	Chemistries Corp.....	13
Influenza Nasal Wash Kit		Randox Laboratories-US, Ltd.....	16	CompuGroup Medical US.....	14	Legionella detection	
SEKISUI Diagnostics.....	16	Interleukin-6 (IL-6)		Labels		Hardy Diagnostics.....	14
Influenza testing		Randox Laboratories-US, Ltd.....	16	CompuGroup Medical US.....	14	Luminex.....	15
DiaSorin Molecular.....	14	Internet-based ordering and reporting		Globe Scientific.....	14	Legionella pneumophila urine antigen	
Luminex.....	15	Clinical Software Solutions.....	13	Laboratory accreditation		Hardy Diagnostics.....	14
SEKISUI Diagnostics.....	16	College of American Pathologists.....	13	College of American Pathologists.....	13	Leukemia	
Iron		Iron		Denline Uniforms.....	14	BD Biosciences.....	12
Carolina Liquid		Carolina Liquid		Laboratory Accreditation Services		Leukemia, subtyping	
Chemistries Corp.....	13	Chemistries Corp.....	13	College of American Pathologists.....	13	BD Biosciences.....	12
SEKISUI Diagnostics.....	16	SEKISUI Diagnostics.....	16				

Page	Page	Page	Page
Leukemia/lymphoma monitor profile	Lithium, serum	Methicillin-resistant staphylococcus aureus (MRSA) culture only Rare	Molecular cytogenetics
BD Biosciences..... 12	Carolina Liquid	Hardy Diagnostics..... 14	College of American
Leukocyte Adhesion Profile (CD11a, Cd11b, CD11c, CD18, CD45)	Chemistries Corp..... 13	Methicillin-resistant Staphylococcus aureus (MRSA), Nucleic Acid Amplification (NAA)	Pathologists 13
Randox Laboratories-US, Ltd. 16	Randox Laboratories-US, Ltd. 16	Luminex..... 15	Molecular diagnostics
Leukocytes	SEKISUI Diagnostics 16	Method development and validation	College of American
BD Biosciences..... 12	Low-density lipoprotein testing	Hardy Diagnostics..... 14	Pathologists 13
Randox Laboratories-US, Ltd. 16	Randox Laboratories-US, Ltd. 16	Microalbumin assays (MALB)	DiaSorin Molecular..... 14
Leukocytes, antibodies	SEKISUI Diagnostics 16	Carolina Liquid	Hardy Diagnostics..... 14
BD Biosciences..... 12	Lp(a)	Chemistries Corp..... 13	Luminex..... 15
Leukocytes, residual, in leukoreduced platelets	Carolina Liquid	Randox Laboratories-US, Ltd. 16	Randox Laboratories-US, Ltd. 16
BD Biosciences..... 12	Chemistries Corp..... 13	SEKISUI Diagnostics 16	Molecular genetics
Lidocaine assays	Randox Laboratories-US, Ltd. 16	Microalbumin assays, controls	College of American
Randox Laboratories-US, Ltd. 16	Lymphocyte antibodies	Randox Laboratories-US, Ltd. 16	Pathologists 13
Linearity materials	BD Biosciences..... 12	Microbes, susceptibility testing (See Drug susceptibility testing)	Luminex..... 15
Bio-Rad Laboratories 13	Lymphocytes, subset enumeration	Hardy Diagnostics..... 14	Molecular pathology
College of American	BD Biosciences..... 12	Microbial strain typing	College of American
Pathologists 13	Lymphoma	Hardy Diagnostics..... 14	Pathologists 13
Randox Laboratories-US, Ltd. 16	BD Biosciences..... 12	Microbiology products and services	Monoclonal antibodies
Lipase	Lymphoma profile, lymph node and solid tissue	College of American	BD Biosciences..... 12
Carolina Liquid	BD Biosciences..... 12	Pathologists 13	Mononucleosis
Chemistries Corp..... 13	Magnesium	Denline Uniforms..... 14	SEKISUI Diagnostics 16
SEKISUI Diagnostics 16	Carolina Liquid	Hardy Diagnostics..... 14	Morphine assay
Lipase assay (LIP)	Chemistries Corp..... 13	Puritan Medical Products..... 16	Randox Laboratories-US, Ltd. 16
Carolina Liquid	SEKISUI Diagnostics 16	Microbiology products and services, automated	MRSA (See Staphylococcus indentification, methicillin resistant)
Chemistries Corp..... 13	Magnesium, serum	Hardy Diagnostics..... 14	Luminex..... 15
Randox Laboratories-US, Ltd. 16	Carolina Liquid	Microsatellite Instability Software	Mycobacteriology
Lipase, serum (LIP)	Chemistries Corp..... 13	SoftGenetics 17	College of American
Carolina Liquid	Marijuana, urine assays (Cannabinoids)	Microscope, lighting	Pathologists 13
Chemistries Corp..... 13	Randox Laboratories-US, Ltd. 16	Globe Scientific 14	Mycology products and services
Randox Laboratories-US, Ltd. 16	Maternal serum, control	Microscopes	College of American
Lipid panel	Randox Laboratories-US, Ltd. 16	Globe Scientific 14	Pathologists 13
Randox Laboratories-US, Ltd. 16	Media, chromogenic	Microscopes, accessories	Mycoplasma pneumoniae
Lipid panel with LDL/HDL ratio	Hardy Diagnostics..... 14	Globe Scientific 14	Luminex..... 15
Randox Laboratories-US, Ltd. 16	Methadone assays	Microscopes, clinical	Mycoplasma pneumoniae culture
Lipid panel with total cholesterol/HDL ratio	Carolina Liquid	Globe Scientific 14	Hardy Diagnostics..... 14
Randox Laboratories-US, Ltd. 16	Chemistries Corp..... 13	Microscopes, digital cameras	Mycoplasma testing
Lipid testing	Randox Laboratories-US, Ltd. 16	Globe Scientific 14	Luminex..... 15
Randox Laboratories-US, Ltd. 16	Methadone confirmation, urine	MLPA Software	Myoglobin
Lipid testing, controls	Randox Laboratories-US, Ltd. 16	SoftGenetics 17	Carolina Liquid
Randox Laboratories-US, Ltd. 16	Methadone screen, urine	Molecular biology, products & services	Chemistries Corp..... 13
Lipids, total, serum	Carolina Liquid	Denline Uniforms..... 14	Randox Laboratories-US, Ltd. 16
Randox Laboratories-US, Ltd. 16	Chemistries Corp..... 13	Puritan Medical Products..... 16	Myoglobin, assays
Lipoprotein (a) (See LP(a))	Methamphetamine assays	Molecular cytogenetics	Randox Laboratories-US, Ltd. 16
Carolina Liquid	Randox Laboratories-US, Ltd. 16	College of American	Myoglobin, controls
Chemistries Corp..... 13	Methamphetamine assays	Pathologists 13	Randox Laboratories-US, Ltd. 16
Randox Laboratories-US, Ltd. 16	Methamphetamine assays	Needle holders	Globe Scientific 14
Lipoproteins, assays	Randox Laboratories-US, Ltd. 16		
Randox Laboratories-US, Ltd. 16			
Lithium (LI)			
Carolina Liquid			
Chemistries Corp..... 13			

Page	Page	Page	Page
Nephelometry, kits Randox Laboratories-US, Ltd. 16	Organism identification, anaerobic bacteria Hardy Diagnostics..... 14 Techcyte 17	Techcyte 17	Phencyclidine (PCP), qualitative, urine Carolina Liquid Chemistries Corp..... 13
Neuroimmunological disorders, autoimmune KRONUS, Inc..... 15	Organism identification, bacteria Hardy Diagnostics..... 14 Techcyte 17	Parasitology products and services, transport systems Techcyte 17	Phenobarbital, assays and therapeutic drug monitoring (PBARB) Carolina Liquid Chemistries Corp..... 13
Neuropathology College of American Pathologists 13	Organism identification, mold Hardy Diagnostics..... 14	Paroxysmal nocturnal hemoglobinuria (PNH) BD Biosciences..... 12	Phenytoin, assays and therapeutic drug monitoring (PHENY) Carolina Liquid Chemistries Corp..... 13
Next Generation Sequencing (NGS) College of American Pathologists 13	Organism identification, mycobacteria Techcyte 17	Parvovirus testing DiaSorin Molecular..... 14	Phlebotomy products and supplies Denline Uniforms..... 14
Next Generation Sequencing Software Orchard Software Corporation 16 SoftGenetics 17	Organism identification, yeast Hardy Diagnostics..... 14 Techcyte 17	Paternity Analysis Software SoftGenetics 17	Phosphorus Carolina Liquid Chemistries Corp..... 13 SEKISUI Diagnostics 16
NMP 22 bladder tumor marker Rare Techcyte 17	Oxycodone screen, urine Carolina Liquid Chemistries Corp..... 13	Pathology Products and Services College of American Pathologists 13 Denline Uniforms..... 14	Phosphorus, assays Carolina Liquid Chemistries Corp..... 13 Randox Laboratories-US, Ltd. 16
Nongynecologic pap testing College of American Pathologists 13	Oxycodone/Oxymorphone Screen, Urine 100 Carolina Liquid Chemistries Corp..... 13	Pathology products and services Denline Uniforms..... 14	Phosphorus, inorganic (PHOS) Carolina Liquid Chemistries Corp..... 13
Norfentanyl Carolina Liquid Chemistries Corp..... 13	Oxycodone/Oxymorphone Screen, Urine 300 Carolina Liquid Chemistries Corp..... 13	Pathology Products and Services Globe Scientific 14 Randox Laboratories-US, Ltd. 16	Phosphorus, urine Carolina Liquid Chemistries Corp..... 13
Nucleic acid amplification products and detection systems (See also Polymerase chain reaction) College of American Pathologists 13 DiaSorin Molecular..... 14 Luminex..... 15	Pain Management and Reports CompuGroup Medical US..... 14	Pathology products and services, anatomic College of American Pathologists 13	Pinworm preparation Techcyte 17
O	Pancreatic Amylase, Serum Carolina Liquid Chemistries Corp..... 13	Pathology products and services, clinical College of American Pathologists 13 Globe Scientific 14 Randox Laboratories-US, Ltd. 16	Pipette tips Globe Scientific 14 Hardy Diagnostics..... 14 Luminex..... 15
Occupational/environmental toxicology Randox Laboratories-US, Ltd. 16	Panels, custom Clinical Software Solutions..... 13	Pathology products and services, equipment Globe Scientific 14 Randox Laboratories-US, Ltd. 16	Pipettes Globe Scientific 14 Hardy Diagnostics..... 14 Luminex..... 15
OEM manufacturing Puritan Medical Products..... 16 Randox Laboratories-US, Ltd. 16	Panels, quality control Bio-Rad Laboratories 13 CompuGroup Medical US..... 14 Randox Laboratories-US, Ltd. 16	Pathology products and services, molecular College of American Pathologists 13	Pipettors Globe Scientific 14 Hardy Diagnostics..... 14
Oncology, products and services Carolina Liquid Chemistries Corp..... 13 Randox Laboratories-US, Ltd. 16	PAP slide stainers Techcyte 17	Patient management information Advanced Data Systems Corporation 12 Clinical Software Solutions 13 College of American Pathologists 13	Platelet antibodies BD Biosciences..... 12
Opiate confirmation, urine Randox Laboratories-US, Ltd. 16	Pap test products College of American Pathologists 13 Techcyte 17	PCP (See Phencyclidine) Carolina Liquid Chemistries Corp..... 13	Platelets BD Biosciences..... 12
Opiate screen, urine Carolina Liquid Chemistries Corp..... 13 Randox Laboratories-US, Ltd. 16	Parainfluenza DiaSorin Molecular..... 14	Pharmacogenomics (PGx) College of American Pathologists 13 Luminex..... 15	Platelets, leukoreduced, residual leukocytes BD Biosciences..... 12
Opiates, assays Carolina Liquid Chemistries Corp..... 13 Randox Laboratories-US, Ltd. 16	Parasite identification College of American Pathologists 13	Pharmacy management information Clinical Software Solutions 13 StaffReady 17	PNH testing BD Biosciences..... 12
Opiates, blood Randox Laboratories-US, Ltd. 16	Parasitology products and services College of American Pathologists 13		POC connectivity Clinical Software Solutions 13 Orchard Software Corporation 16

Page	Page	Page	Page
Point-of-care testing	Protein, total	R	Recapping sample tube
College of American	Carolina Liquid		LGP (Laboratory Growth &
Pathologists 13	Chemistries Corp. 13		Productivity) 15
Polymerase chain reaction	Randox Laboratories-US, Ltd. 16	Radiology management	Red blood cell (RBC) count
College of American	Publications	information system	Techcyte 17
Pathologists 13	College of American	Clinical Software Solutions 13	Reference bacterial culture
Luminex 15	Pathologists 13	Reagent reservoirs	identification
Potassium testing (K)		Globe Scientific 14	Techcyte 17
(Electrolytes, analysis)		Hardy Diagnostics 14	Reference books
Carolina Liquid	Q	Reagents	College of American
Chemistries Corp. 13	Q fever	Carolina Liquid	Pathologists 13
Potassium testing (See	DiaSorin Molecular 14	Chemistries Corp. 13	Reference materials
also Electrolytes, analysis),	Quality assurance/quality	Hardy Diagnostics 14	Luminex 15
standard	control	Luminex 15	Renal function panel
Randox Laboratories-US, Ltd. 16	Bio-Rad Laboratories 13	Randox Laboratories-US, Ltd. 16	Randox Laboratories-US, Ltd. 16
Potassium, serum	Clinical Software Solutions 13	Reagents - coagulation	Reproductive hormones
Carolina Liquid	Denline Uniforms 14	assay	College of American
Chemistries Corp. 13	Quality assurance/quality	Hardy Diagnostics 14	Pathologists 13
Potassium, urine	control, blood bank	Randox Laboratories-US, Ltd. 16	Respiratory syncytial virus
Carolina Liquid	Bio-Rad Laboratories 13	Reagents, bulk	by PCR
Chemistries Corp. 13	Clinical Software Solutions 13	Carolina Liquid	DiaSorin Molecular 14
Prealbumin, nephelometry	Randox Laboratories-US, Ltd. 16	Chemistries Corp. 13	Luminex 15
kit	Quality assurance/quality	Randox Laboratories-US, Ltd. 16	Respiratory syncytial virus
Randox Laboratories-US, Ltd. 16	control, controls	Reagents, chemistry	testing
Pregnancy test, urine	Bio-Rad Laboratories 13	Carolina Liquid	Luminex 15
Randox Laboratories-US, Ltd. 16	Clinical Software Solutions 13	Chemistries Corp. 13	Respiratory Virus Pan by
Pregnancy testing	Randox Laboratories-US, Ltd. 16	Randox Laboratories-US, Ltd. 16	RT-PCR
Randox Laboratories-US, Ltd. 16	Quality assurance/quality	Reagents, custom	Luminex 15
Primidone, assays and	control, molecular biology	Randox Laboratories-US, Ltd. 16	Respiratory Virus Panel
therapeutic drug monitoring	Bio-Rad Laboratories 13	Reagents, diagnostic	(RVP), PCR
Carolina Liquid	Clinical Software Solutions 13	Carolina Liquid	Luminex 15
Chemistries Corp. 13	Quality assurance/quality	Chemistries Corp. 13	Respiratory virus tests
Procaïnamide, assay and	control, products and	Randox Laboratories-US, Ltd. 16	Luminex 15
therapeutic drug monitoring	services	Reagents, ELISA	Reticulocyte
Carolina Liquid	Bio-Rad Laboratories 13	KRONUS, Inc. 15	BD Biosciences 12
Chemistries Corp. 13	Clinical Software Solutions 13	Randox Laboratories-US, Ltd. 16	Reticulocyte count
Procalcitonin (PCT) Assay	Randox Laboratories-US, Ltd. 16	Reagents, immunoassay	BD Biosciences 12
Carolina Liquid	WSLH Proficiency Testing 17	KRONUS, Inc. 15	Reticulocyte count by flow
Chemistries Corp. 13	Quality assurance/quality	Randox Laboratories-US, Ltd. 16	cytometry
Proficiency testing	control, software	Reagents, instrument-	BD Biosciences 12
College of American	Bio-Rad Laboratories 13	specific, Cobas Mira	Reticulocyte, analysis
Pathologists 13	Clinical Software Solutions 13	Analyzers	BD Biosciences 12
WSLH Proficiency Testing 17	CompuGroup Medical US 14	Randox Laboratories-US, Ltd. 16	Reticulocyte, enumeration
Prolactin assays	Orchard Software Corporation 16	Reagents, instrument-	BD Biosciences 12
Randox Laboratories-US, Ltd. 16	Randox Laboratories-US, Ltd. 16	specific, Hitachi	Rheumatoid factor assays
Propoxyphene screen, urine	StaffReady 17	Randox Laboratories-US, Ltd. 16	(RF)
Carolina Liquid	Quality Management Tools	Reagents, lipid testing	Carolina Liquid
Chemistries Corp. 13	Clinical Software Solutions 13	Randox Laboratories-US, Ltd. 16	Chemistries Corp. 13
Prostate-specific antigen	College of American	Reagents, lipid testing	Randox Laboratories-US, Ltd. 16
Carolina Liquid	Pathologists 13	system	Rickettsia testing
Chemistries Corp. 13	CompuGroup Medical US 14	Randox Laboratories-US, Ltd. 16	DiaSorin Molecular 14
Protein, controls	StaffReady 17	Reagents, stable liquid	Rotavirus detection
Randox Laboratories-US, Ltd. 16	WSLH Proficiency Testing 17	Randox Laboratories-US, Ltd. 16	Luminex 15
Protein, standard	Quality System Management	Reagents, therapeutic drug	RSV
Randox Laboratories-US, Ltd. 16	Clinical Software Solutions 13	monitoring	Luminex 15
	College of American	Randox Laboratories-US, Ltd. 16	
	Pathologists 13	Reagents, virology	
		Randox Laboratories-US, Ltd. 16	

Page	Page	Page	Page
RSV (Respiratory syncytial virus) Luminex..... 15	Serum, specialty controls Randox Laboratories-US, Ltd..... 16	Staphylococcus identification, methicillin resistant Hardy Diagnostics..... 14	T-cell counts BD Biosciences..... 12
S	Services Clinical Software Solutions..... 13	Stem cell (CD34+) count BD Biosciences..... 12	T-cell subset functional responses, assessment BD Biosciences..... 12
Safety products Denline Uniforms..... 14 Hardy Diagnostics..... 14 LGP (Laboratory Growth & Productivity)..... 15	Shigella identification Hardy Diagnostics..... 14 Luminex..... 15	Stem cell detection BD Biosciences..... 12	TDM (See Therapeutic drug monitoring) College of American Pathologists..... 13 Randox Laboratories-US, Ltd..... 16
Safety Products, Disposable Denline Uniforms..... 14 Hardy Diagnostics..... 14	Slide stainers Hardy Diagnostics..... 14	Sterile production service Puritan Medical Products..... 16	Test kits, rapid Hardy Diagnostics..... 14
Safety Products, Lab Coats Denline Uniforms..... 14 Hardy Diagnostics..... 14	Sodium testing (NA) (Electrolytes, analysis) Carolina Liquid Chemistries Corp..... 13	Stool culture, comprehensive Techcyte..... 17	Test Tube Racks Globe Scientific..... 14
Salicylate, serum Carolina Liquid Chemistries Corp..... 13	Sodium, serum Carolina Liquid Chemistries Corp..... 13	Stool, occult blood Hardy Diagnostics..... 14	Testosterone, assays Carolina Liquid Chemistries Corp..... 13
Salicylates, assays Carolina Liquid Chemistries Corp..... 13 SEKISUI Diagnostics..... 16	Sodium, urine Carolina Liquid Chemistries Corp..... 13	Storage equipment Globe Scientific..... 14	Theophylline, assay and therapeutic drug monitoring (THEO) Carolina Liquid Chemistries Corp..... 13
Salmonella identification Hardy Diagnostics..... 14 Luminex..... 15	Software connectivity Clinical Software Solutions..... 13 CompuGroup Medical US..... 14	Strep A antigen SEKISUI Diagnostics..... 16	Theophylline, serum Carolina Liquid Chemistries Corp..... 13
Sample management systems Clinical Software Solutions..... 13	Sorbitol Dehydrogenase (SDH) SEKISUI Diagnostics..... 16	Strep A, Ultra SEKISUI Diagnostics..... 16	Therapeutic drug monitoring Carolina Liquid Chemistries Corp..... 13 College of American Pathologists..... 13
SARS Associated Coronavirus by RT-PCR College of American Pathologists..... 13 Luminex..... 15	Specimen management systems Clinical Software Solutions..... 13 College of American Pathologists..... 13	Streptococcus - group A Hardy Diagnostics..... 14 Luminex..... 15	Therapeutic drug monitoring, controls Randox Laboratories-US, Ltd..... 16
SARS coronavirus RNA, PCR College of American Pathologists..... 13 Luminex..... 15	Specimen, collection Denline Uniforms..... 14 Globe Scientific..... 14 Puritan Medical Products..... 16	Streptococcus (group B) culture, genital Hardy Diagnostics..... 14	Therapeutic drug monitoring, reagents Randox Laboratories-US, Ltd..... 16
SARS-CoV-2 - Antibody Test College of American Pathologists..... 13 Hardy Diagnostics..... 14 Luminex..... 15	Specimen, transport Globe Scientific..... 14 Puritan Medical Products..... 16	Streptococcus identification Hardy Diagnostics..... 14	Thyroglobulin, test kits Randox Laboratories-US, Ltd..... 16
Seminars College of American Pathologists..... 13	Specimens Puritan Medical Products..... 16	Streptococcus pneumoniae antigen Hardy Diagnostics..... 14	Thyroid function Randox Laboratories-US, Ltd..... 16
Sequence Analysis Software SoftGenetics..... 17	Specimens, storage equipment Globe Scientific..... 14	Streptococcus pneumoniae testing Hardy Diagnostics..... 14 Luminex..... 15	Thyroid hormone assays Carolina Liquid Chemistries Corp..... 13
Serology products and services Randox Laboratories-US, Ltd..... 16	Specimens, transport systems Puritan Medical Products..... 16	Surgical pathology College of American Pathologists..... 13	Thyroid panel Randox Laboratories-US, Ltd..... 16
Serum proteins Randox Laboratories-US, Ltd..... 16	Staining and Coverslipping Globe Scientific..... 14	Susceptibility testing College of American Pathologists..... 13 Hardy Diagnostics..... 14	Thyroid panel with TSH Randox Laboratories-US, Ltd..... 16
Serum proteins, calibrators Randox Laboratories-US, Ltd..... 16	Stains Hardy Diagnostics..... 14	Syphilis, diagnosis Randox Laboratories-US, Ltd..... 16	Thyroid peroxidase, test kits Randox Laboratories-US, Ltd..... 16
Serum proteins, controls Randox Laboratories-US, Ltd..... 16	Stains, cytology Techcyte..... 17	T	Thyroid-stimulating hormone (TSH) Carolina Liquid Chemistries Corp..... 13 Randox Laboratories-US, Ltd..... 16
	Standards Randox Laboratories-US, Ltd..... 16	T3 Randox Laboratories-US, Ltd..... 16	Thyroid-stimulating hormone receptor antibody tests KRONUS, Inc..... 15
	Staph mec A Luminex..... 15	T4 Carolina Liquid Chemistries Corp..... 13 Randox Laboratories-US, Ltd..... 16	
		T-cell activation profile, CD8 subsets BD Biosciences..... 12	

Page	Page	Page	Page
Thyroxine (See also Thyroid hormone assays)	Triglyceride (TRIG)	Uric acid (UA)	Urine, ELISA kits
Carolina Liquid	Carolina Liquid	Carolina Liquid	Randox Laboratories-US, Ltd. 16
Chemistries Corp. 13	Chemistries Corp. 13	Chemistries Corp. 13	Urine, human, control
Thyroxine (T4)	Randox Laboratories-US, Ltd. 16	Randox Laboratories-US, Ltd. 16	Randox Laboratories-US, Ltd. 16
Carolina Liquid	Triglyceride, assay	SEKISUI Diagnostics 16	Urine, sediment analysis
Chemistries Corp. 13	Carolina Liquid	Uric acid, assays	Techcyte 17
TIBC (See Iron-binding capacity)	Chemistries Corp. 13	Carolina Liquid	Urine, specialty controls
Carolina Liquid	Triglyceride, liquid	Chemistries Corp. 13	Randox Laboratories-US, Ltd. 16
Chemistries Corp. 13	Carolina Liquid	Uric acid, serum	Urine, specimen transport
T-lymphocyte helper/suppressor profile	Chemistries Corp. 13	Carolina Liquid	Globe Scientific 14
BD Biosciences 12	Randox Laboratories-US, Ltd. 16	Chemistries Corp. 13	Urine, test strips
Tobramycin, assay and therapeutic drug monitoring (TOB)	SEKISUI Diagnostics 16	Urinalysis	Globe Scientific 14
Carolina Liquid	Triiodothyronine (T3)	College of American	UroVysionFISH
Chemistries Corp. 13	Randox Laboratories-US, Ltd. 16	Pathologists 13	Techcyte 17
Total Bilirubin (TBIL)	Triiodothyronine (T3), free, serum	Hardy Diagnostics 14	UroVysionfor bladder cancer
Carolina Liquid	Randox Laboratories-US, Ltd. 16	Randox Laboratories-US, Ltd. 16	Techcyte 17
Chemistries Corp. 13	Trisomy/Aneuploidy Software	Urinalysis, controls	
Total IgE	SoftGenetics 17	Randox Laboratories-US, Ltd. 16	
Randox Laboratories-US, Ltd. 16	Troponin I	Urinalysis, specimen transport	
Total Protein (TP)	Randox Laboratories-US, Ltd. 16	Globe Scientific 14	
Carolina Liquid	Troponin testing	Urine	
Chemistries Corp. 13	Randox Laboratories-US, Ltd. 16	Randox Laboratories-US, Ltd. 16	
Toxicology products and services	TSH (See Thyroid-stimulating hormone)	Urine chemistries, amylase	
Carolina Liquid	Randox Laboratories-US, Ltd. 16	Randox Laboratories-US, Ltd. 16	
Chemistries Corp. 13	Tumor markers	Urine chemistries, BUN	
Clinical Software Solutions 13	College of American	Randox Laboratories-US, Ltd. 16	
College of American	Pathologists 13	Urine chemistries, calcium	
Pathologists 13	Randox Laboratories-US, Ltd. 16	Randox Laboratories-US, Ltd. 16	
CompuGroup Medical US 14	Tumor necrosis factor alfa	Urine chemistries, chloride	
Globe Scientific 14	Randox Laboratories-US, Ltd. 16	Randox Laboratories-US, Ltd. 16	
Randox Laboratories-US, Ltd. 16	T-uptake test (TUP)	Urine chemistries, creatinine	
Training	Carolina Liquid	Randox Laboratories-US, Ltd. 16	
Clinical Software Solutions 13	Chemistries Corp. 13	Urine chemistries, glucose	
WSLH Proficiency Testing 17		Randox Laboratories-US, Ltd. 16	
Transferrin (TRF)		Urine chemistries, magnesium	
Carolina Liquid		Randox Laboratories-US, Ltd. 16	
Chemistries Corp. 13		Urine chemistries, phosphorus	
Randox Laboratories-US, Ltd. 16		Randox Laboratories-US, Ltd. 16	
Transferrin assays (TRF)		Urine chemistries, potassium	
Randox Laboratories-US, Ltd. 16		Randox Laboratories-US, Ltd. 16	
Transfusion medicine		Urine chemistries, sodium	
College of American		Randox Laboratories-US, Ltd. 16	
Pathologists 13		Urine chemistries, urea	
Denline Uniforms 14		Randox Laboratories-US, Ltd. 16	
Trichomonas vaginalis, rapid testing		Urine chemistries, uric acid	
SEKISUI Diagnostics 16		Randox Laboratories-US, Ltd. 16	
Tricyclic antidepressants, serum		Urine cytology	
Carolina Liquid		Techcyte 17	
Chemistries Corp. 13		Urine, analyzers	
		Randox Laboratories-US, Ltd. 16	
		Urine, chemistry, controls	
		Randox Laboratories-US, Ltd. 16	

West Nile virus

Diasorin Molecular..... 14

Western blot technology

Randox Laboratories-US, Ltd..... 16

White blood cell (WBC) count

Techcyte..... 17

White blood cell residual in leukoreduced blood products

BD Biosciences..... 12

Workshops

College of American Pathologists 13

Luminex..... 15

Z

ZAP-70 in B-CLL

BD Biosciences..... 12

Zinc, serum

Randox Laboratories-US, Ltd..... 16

Zinc, urine


Randox Laboratories-US, Ltd..... 16

INDEX OF ADVERTISERS

Advertiser WebPage

College of American Pathologists.....	cap.org.....	C2
Diasorin Molecular	molecular.diasorin.com.....	C3
Health Care Logistics.....	GoHCL.com	8
Kronus	kronus.com.....	3
Randox Laboratories	randox.com/rxseries.....	11
Sekisui Diagnostics.....	osomtests.com.....	C4

This index is provided as a service. The publisher does not assume liability for errors or omissions.

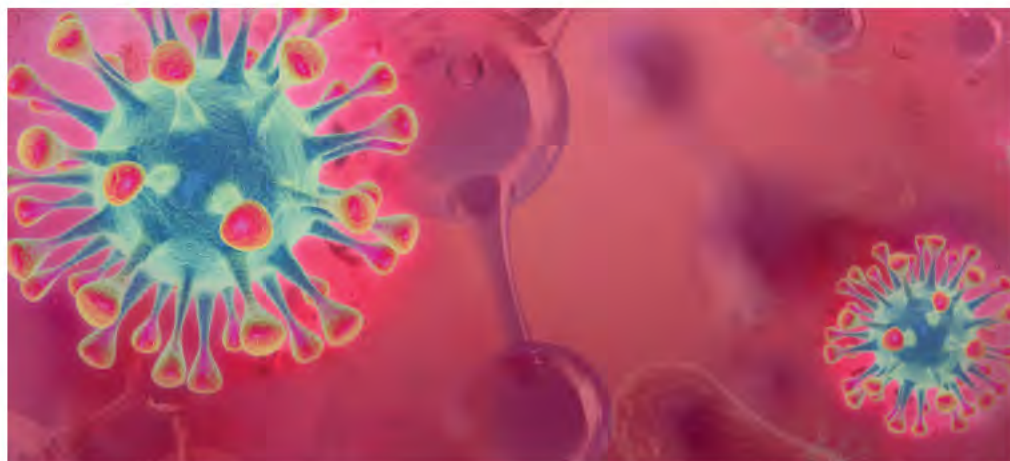

Peer-Reviewed Insight for Clinical Laboratory Professionals At Your Fingertips



Complimentary Subscription

@ MLO-ONLINE.COM/SUBSCRIBE



Print



eNews



Digital Edition



COMPLETE YOUR CONTINUING EDUCATION HOURS **ONLINE!**



Take the monthly test online for immediate certification or download the printable test from www.mlo-online.com/ce (Mail-in tests may take up to 6 weeks to score.)



The CE articles and accompanying CE tests remain online until the indicated deadlines have passed.

Completed tests and fees go directly to Northern Illinois University (NIU), and scores of 70% and higher qualify for one contact hour of P.A.C.E. credit.

Easy registration and payment options are available through NIU by following the link found at www.mlo-online.com/ce

MLO and NIU DeKalb, IL, are co-sponsors in offering continuing education units (CEUs) for each issue's CE article. CEUs or contact hours are granted by the College of Health and Human Sciences at Northern Illinois University, which has been approved as a provider of continuing education programs in the clinical laboratory sciences by the ASCLS P.A.C.E.® program. Approval as a provider of continuing education programs has been granted by the state of Florida (Provider No. JP0000496). Continuing education credits awarded for successful completion of this test are acceptable for the ASCP Board of Registry Continuing Competence Recognition Program.



NORTHERN ILLINOIS UNIVERSITY

College of Health and Human Sciences



Simplexa[®] Congenital CMV Direct Kit

Direct detection of cytomegalovirus DNA in
saliva swab and urine specimens

- **Broad coverage:**
Validated for both saliva and confirmatory urine specimens from infants less than 21 days of age.
- **Sample-to-answer workflow with timely results:**
CLIA moderate complexity providing results in about an hour - No DNA extraction required.
- **High performance:**
Confirmatory urine testing (prospective and retrospective combined) 99.9% Negative Percent Agreement.

Request more information:
<https://bit.ly/cCMVRequest>

molecular.diasorin.com/us/kit/simplexa-congenital-cmv-direct-kit/
+1 (562) 240-6500



 **DiaSorin
Molecular**

The quality of treatment starts with diagnosis

PRTA596451.0623

Simplexa, LIAISON MDX, and the associated logos are all trademarks or registered trademarks of DiaSorin Molecular LLC, or its affiliate, in the U.S. and/or other countries. Product availability subject to required regulatory approvals.



Resilience + Passion

We make diagnostics that **matter**

We recognize your **passion** for providing high quality care to patients displaying symptoms associated with respiratory infections, and we appreciate your efforts and **resiliency** working to reduce the number of people impacted by their spontaneous and rapid spread.

At SEKISUI Diagnostics we are committed to providing **high quality, point-of-care tests for detecting the most common causes of respiratory infections**, so you can get the answers fast and your patients back to doing what they love.

Like you, we understand there is a patient behind every answer—and that's what matters most.

Learn more about our broad range of OSOM® respiratory rapid tests including COVID-19, Flu and Strep A.



For more information, call
800-332-1042, or scan QR code



SEKISUI
DIAGNOSTICS
Because every result matters™